

MagNA Pure 24 System protocols

| Target | Protocol name | Input sample type and sample volumes | | | | | | | | | | Elution volume | Run time [†] (h:mm) |
|---------------------------------|---|--|--------|----------------------|--------------|-------|-------|------------------------------|----------------|--|----------|----------------|------------------------------|
| | | Whole blood | Plasma | Serum | Nasal swabs* | BAL** | Urine | Stool | Cultured cells | Fresh-frozen tissue | FFPET*** | | |
| Bacterial, fungal, and viral NA | Pathogen 200 Protocol optimized for yield and purity | 200 µL | | | | | | | | | | 50 or 100 µL | 1:14 |
| | Fast Pathogen 200 8 sample protocol optimized for speed | | | | | | | | | | | | 0:33 |
| | Pathogen 1000 Protocol optimized for yield and purity | 500 or 1,000 µL | | | | | | | | | | | 1:31 |
| | External Lysis Pathogen 200 Protocols optimized with external lysis buffer | 450 µL lysate from 200 µL sample | | | | | | | | | | | 1:22 |
| | External Lysis Pathogen 500 Protocols optimized with external lysis buffer | 1,450 µL lysate from 500 µL sample | | | | | | | | | | | 1:31 |
| Genomic DNA | hgDNA 200 Protocol optimized for human genomic NA | | | | | | | | | Up to 5 mg | | 50 or 100 µL | 1:07 |
| | Fast hgDNA 200 8 sample protocol optimized for speed | 200 µL (≤2 × 10 ⁶ cells) | | | | | | Up to 5 × 10 ⁵ | | | | | 0:27 |
| | hgDNA ds 200 Recommended when double-stranded DNA is required | | | | | | | | | | | | 1:07 |
| | hgDNA 1000 Protocol optimized for human genomic NA | 500 µL (≤5 × 10 ⁶ cells) 1,000 µL (≤1 × 10 ⁷ cells) | | | | | | Up to 1 × 10 ⁶ | | | | 100 or 200 µL | 2:16 |
| | DNA FFPET 1000 Deparaffinization and lysis on board | | | | | | | | | Up to 6 × 5 µm sections (up to 6 mm ²) | | 50 or 100 µL | 5:36 |
| Cell-free NA | cfNA ss 2000 cfNA ss 4000 Protocols optimized for single-stranded DNA | | | 2,000 µL 4,000 µL | | | | | | | | 50 or 100 µL | 1:44 |
| | cfNA ds 2000 cfNA ds 4000 Protocols optimized for double-stranded DNA | | | 2,000 µL 4,000 µL | | | | | | | | | 100, 150, or 200 µL |
| | cfNA ds 4000 hp Protocol optimized for double-stranded DNA and NGS | | | 4,000 µL | | | | | | | | 60 or 150 µL | 2:49 |
| | cfNA ds 4000 hp Protocol optimized for double-stranded DNA and NGS | | | 4,000 µL | | | | | | | | 60 or 150 µL | 3:57 |
| Liquid handling | Sample Transfer to the processing cartridge for 24 samples | | | | | | | 200 µL 500 µL 1,000 µL | | | | 50 or 100 µL | 0:08 |
| | PCR Setup for 2-25 µL per eluate for PCR Setup in FrameStrip with flat caps-Low Profile or LightCycler® 8-tube strips for 24 eluates | | | | | | | | | | | | 0:01 |
| | Archiving for 25-50 µL per eluate FrameStrip low profile strips, or LightCycler® 8-tube strips for 24 eluates | | | | | | | | | | | | 0:01 |

*Nasopharyngeal/nasal swabs

**BAL - Bronchoalveolar lavage

***FFPET - formalin-fixed paraffin-embedded tissue

[†]These times are approximate and should only be used as a guidance.

Start here. Go Anywhere.

All MagNA Pure 24 kits, consumables, and accessories are for in vitro diagnostic unless otherwise noted.
MAGNA PURE and LIGHTCYCLER are registered trademarks of Roche.
All other product names and trademarks are the property of their respective owners.

©2024 Roche Molecular Systems, Inc. All rights reserved.

Published by
Roche Sequencing and Life Science
9115 Hague Road
Indianapolis, IN 46256

MC-US-05971 02/24