CoaguChek[®] Vantus System

User's Manual

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© 2018-2019 Roche Diagnostics GmbH

The contents of this document, including all graphics, are the property of Roche Diagnostics. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Roche Diagnostics. Roche Diagnostics has made every reasonable effort to ensure that all the information contained in this manual is correct at the time of printing. However, Roche Diagnostics reserves the right to make any changes necessary without notice as part of ongoing product development.

Please send questions or comments about this manual to your local Roche representative.

COAGUCHEK, COAGUCHEK VANTUS, and SOFTCLIX are trademarks of Roche.

CONTINUA, the CONTINUA Logos and CONTINUA CERTIFIED are trademarks, service marks, or certification marks of the Continua Health Alliance. CONTINUA is a registered trademark in some, but not all countries this product is distributed.

The *Bluetooth*[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license. Other trademarks and product names are the property of their respective owners.

Request additional copies of this document by sending an email to indianapolis.rsn_documentation@roche.com. Request the document number OS-00425-02.

Revision History

Version	Date	Content
1.0	2018-01	New document
2.0	2018-07	Added warranty information, minor revisions
3.0	2019-08	Update to SW 01.05: added information about BLE device address; minor revisions

What is new in publication version 3.0?

SW01.05.xx.....	9
Handling changes.....	9
Editorial revisions.....	9

Introduction

Before you start.....	11
Intended use.....	11
PT time.....	12
Self-testing	12
The CoaguChek Vantus System.....	14
Test principle	16
Operating conditions.....	18
Quality control.....	19
Important Safety Instructions and Additional Information	20
The CoaguChek Vantus Meter.....	22
Overview of the meter elements.....	22
Batteries	24

Starting Up

Inserting the batteries.....	28
Initially setting up the meter.....	31
Code chip.....	36
Inserting the code chip.....	37

Testing a Capillary Blood Sample

Important notes	40
Getting a good capillary blood sample.....	42
Preparing to test.....	43
Performing a test.....	44
Notes concerning the results display.....	55

Reviewing and Handling Data

Reviewing test results.....	58
Displaying reports on the meter.....	60
Displaying a Trend report	61
Displaying a Target report	64
Transferring data	66
Data transfer via USB.....	67
Data transfer via <i>Bluetooth</i>	69
Displaying reports on a computer	71
Security settings in the Internet browser.....	73
Virus scanner.....	73
Reports overview.....	74
Time period	75
Trend graph.....	75
Target range display.....	76
Target graph.....	76
List of results	77
Printing reports.....	78
Using data in external applications.....	79

Reminders and Meter Settings

Setting up reminders	81
New reminder	82
Editing or deleting a reminder	86
Meter settings	87
Settings overview	88
Language.....	90
Time & Date	91
Result Format	93
Set Targets	94
Beeper Volume.....	97
Display Brightness.....	98
Button Click	99
Data Transfer	100

Cleaning and Disinfecting the Meter and the Lancing Device

Overview.....	103
What is the difference between cleaning and disinfecting?	104
When to clean and disinfect the meter and the lancing device	104
Approved cleaning and disinfecting product.....	105
Cleaning the meter housing	106
Cleaning the test strip guide and test strip guide cover.....	108
Disinfecting the meter	111
Disinfecting the test strip guide cover and test strip guide.....	112
Cleaning and Disinfecting the Lancing Device	117
When to clean and disinfect the lancing device.....	117
What to clean and disinfect	119
How to clean and disinfect the lancing device	119

Other Functions and Troubleshooting

Other functions	121
About (manufacturing information and data encryption key)	122
Clear Memory	123
Meter Log	125
Display Test	126
Firmware Update	127
Troubleshooting	129
Errors and unusual behavior without error messages	130

Symbols, Icons, Messages

Symbols on packaging and meter	131
Icons on the display	133
Messages	135
Reminders	136
Information	138
Maintenance	140
Warnings	144
Error messages	145

General Product Information

Ordering	149
Product limitations.....	149
Operating conditions and technical data	150
Sample material.....	151
Storage and transport conditions.....	151
Disposal of the CoaguChek Vantus meter	152
For healthcare professionals	153
Disposal of used batteries	153
Repairs.....	153
Information service.....	154
Product Safety Information.....	155
<i>Bluetooth</i> Wireless Technology	155
Radio Frequency Communication	156
Electromagnetic Compatibility	157
Limited One (1) Year Warranty.....	158

Index

This page intentionally left blank.

What is new in publication version 3.0?

This section provides an overview of all major changes from User's Manual version 2 to version 3. Deletions or minor corrections are not listed.

SW01.05.xx

- Update to the *About* screen (BLE device address). See page 122.
- Updated maintenance message [M-28]. See page 141.

Handling changes

- Updated description of the control bottle icon. See page 51.
- Additional safety related information for data transfer to a computer. See pages 72 and 73.

Editorial revisions

- Minor editorial revisions.

This page intentionally left blank.

Before you start

Intended use

The CoaguChek[®] Vantus System measures an INR (International Normalized Ratio) based on a prothrombin time (PT) response to monitor the effect of a therapy with vitamin K antagonists by using the CoaguChek XS[®] PT Test strips. The CoaguChek Vantus System uses fresh capillary whole blood from a finger stick.

The system is intended for properly selected and suitable trained users on the prescription of the treating doctor.

Users should be stabilized on anticoagulation with vitamin K antagonists for at least 6 weeks prior to single patient self-testing with the CoaguChek Vantus System.

The CoaguChek Vantus System is intended for single patient self-testing only for adults, age 22 years and older.

PT time

A PT test is often called an INR test. The International Normalized Ratio (INR) is a way of standardizing the results of prothrombin time tests because there are different test methods. INR enables you and your doctor to understand your coagulation results even when they come from different labs using different test methods.

Self-testing

If you self-test, you need to discuss your therapy model with your healthcare professional first. Self-testing is not a substitute for regular visits to your doctor.

Depending on the training you have received and the instructions from your healthcare professional, you might communicate your results to a healthcare professional or clinic to receive the appropriate dose adjustment. In any event, you should follow the instructions you have been given regarding how to report your test results.

Self-monitoring of oral anticoagulation therapy using coagulation values determined with the CoaguChek Vantus System may only be undertaken after consultation with your doctor and comprehensive instruction by a qualified healthcare professional.

This manual contains all the information you need to operate and care for the CoaguChek Vantus System. Please **read this entire manual carefully** before you use the meter.

The CoaguChek Vantus System

The CoaguChek Vantus System makes coagulation testing easy. The graphical user interface of the CoaguChek Vantus meter guides you through the test step by step. The code chip accompanying the test strips contains strip lot-specific information and the expiry date for the test strips. You only need to insert the code chip, turn the meter on, insert the test strip, and apply a blood sample. The CoaguChek Vantus meter measures your coagulation time and displays the result. After the test is completed, the meter automatically saves the result to memory.

If you have any questions about the CoaguChek Vantus meter, please contact CoaguChek Technical Services at 1-800-428-4674. You will find further contact details on page 154.

Please note: Before you use the meter for the first time (i.e. after you have first inserted the batteries), you must set the date and time correctly to allow you to carry out measurements properly. Each time you replace the batteries, you need to check (and, if necessary, adjust) the date and time.

Note on screenshots:
The test results and software and hardware information shown in the screenshots in this manual are for illustration purposes only. The information displayed on your meter may differ.

Test principle

The CoaguChek XS PT Test test strip contains a lyophilized reagent (reagent in dried form). The reactive components of this reagent consist of thromboplastin and a peptide substrate. When a sample is applied, thromboplastin activates coagulation, which leads to the formation of thrombin. At the same time, the meter starts to measure the time. The enzyme thrombin cleaves the peptide substrate, generating an electrochemical signal. Depending on the time elapsed when it first appears, this signal is then converted by means of an algorithm into an INR result.

Contents of the pack

- CoaguChek Vantus meter
- 4 alkaline batteries, 1.5 V, type AAA (LR03)¹
- USB cable
- CoaguChek XS Softclix lancing device with instructions for use
- CoaguChek Softclix Lancet
- System carry-case (not shown here)
- User's Manual (this document)
- Quick Reference Guide (not shown here)

¹ also available as "alkaline-manganese dioxide batteries"

Operating conditions

To ensure that your CoaguChek Vantus System functions properly, please observe the following guidelines:

- Only use the meter at a temperature between 59 °F and 90 °F.
- Only use the meter at a relative humidity between 10% and 85%.
- When testing, place the meter on a level, vibration-free surface or hold it so that it is roughly horizontal.
- If the meter is to remain unused for a longer period of time, keep it in the carry-case supplied.
- Use the meter only up to a maximum altitude of 13120 feet.

Electromagnetic interference

Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use the meter near strong electromagnetic fields.

Quality control

The CoaguChek Vantus System has a number of built-in quality control functions such as:

A check of the electronic components and functions every time the meter is turned on.

A check of the test strip temperature while a test is in progress.

A check of the expiry date and lot information on the test strip.

A quality control function is incorporated into the test strip. These on-board quality control checks verify the test strip integrity.

Important Safety Instructions and Additional Information

This section explains how safety-related messages and information related to the proper handling of the system are presented in the CoaguChek Vantus User's Manual. Read these passages carefully.

The safety alert symbol alone (without a signal word) promotes awareness to hazards which are generic or directs the reader to related safety information.

WARNING

Indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

NOTICE

Indicates a hazardous situation which, if not avoided, may result in damage to the system.

Important information that is not safety relevant is presented against a colored background (without a symbol). Here you will find additional information on correct use of the meter or useful tips.

The CoaguChek Vantus Meter

Overview of the meter elements

Overview of the meter elements (front)

A Display

Shows menus, results, messages and results recalled from memory.

B Back button ↶

Exit the currently displayed menu.

C Up/down arrows button ▲▼

Move or scroll up/down in a screen.

D Enter button (Power on) ◼

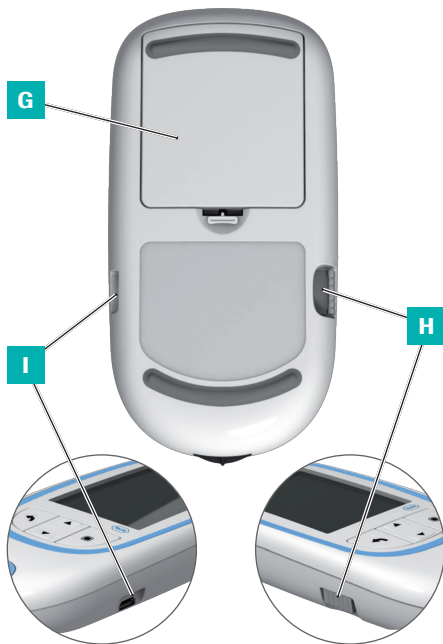
Confirm the currently selected option or perform a function. Also used to power the meter on and off.

E Test strip guide cover

Remove this cover to clean the test strip guide.

F Test strip guide

Insert the test strip here.

Overview of the meter elements (back and sides)

G Battery compartment cover

Covers the battery compartment (four 1.5 V alkaline batteries type AAA (LR03)).

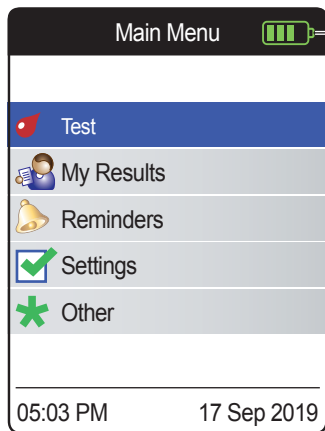
H Code chip slot

Insert the code chip here.

I USB port

For wired communication and data transfer to a computer.

Batteries

To save power, the CoaguChek Vantus meter automatically turns itself off after 2 minutes unless a button has been pressed or a new test strip has been inserted. When the meter turns itself off, all results obtained up to that point remain in memory.

When the **Main Menu** screen is displayed, the meter indicates the battery power level on the top right of the display. The battery symbol is divided into four segments which correspond to the battery power level. Replace batteries at latest when the flashing red battery symbol or a corresponding message appears.

Please be aware that battery life may be affected by many factors, such as battery type, quality, operating conditions (e.g. ambient temperature), frequency of use, and test duration.

The meter retains all data in the memory, even when no batteries are inserted. This includes, in particular, your test results and all settings.

However, if the meter is without batteries for more than 30 minutes, the current time (clock time) and date may be lost. In this case, you might have to re-enter the time and date manually.

Think of the environment. Dispose of used batteries sensibly.

**WARNING**

Do not throw batteries onto a fire. They may explode!

This page intentionally left blank.

Starting Up

Before using the meter for the first time, carry out the following steps:

- Insert the batteries
- Set the language
- Set the current date and time
- Insert the code chip (you can also carry out this step immediately before testing)

Inserting the batteries

When replacing the batteries, you should insert the new batteries within 30 minutes of removing the old ones, to keep the date and time settings. If you take longer, you may have to re-enter the date and time. Only use alkaline batteries 1.5 V type AAA (LR03).¹

With normal use, the batteries will last for approximately 60 tests.

Roche recommends replacing used batteries only with new high quality batteries and not mixing different brands. Do not mix new and already used batteries.

Think of the environment. Dispose of used batteries sensibly.

¹ also available as "alkaline-manganese dioxide batteries"

Use of rechargeable batteries is not recommended. If you do use rechargeable batteries, the following may happen:

- The number of tests that can be performed per battery charge is much lower than with alkaline batteries.
- Performing tests in a cold environment may not be possible.
- The capacity remaining may not be displayed correctly. If the meter displays a Battery Low message, performing further tests may be impossible. Replace the batteries as soon as possible.

- 1 Turn the meter over.
- 2 Gently press the tab on the battery compartment cover towards the top of the meter and lift off the cover.
- 3 Insert the four batteries into the battery compartment as indicated. Pay attention to the positions of “+” (top of battery) and “-” (flat end).
- 4 Re-attach the cover. Close the battery compartment by pushing the cover down until the tab snaps into place.

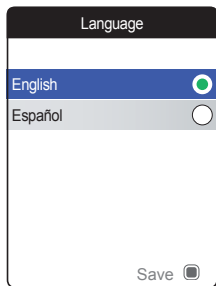
Initially setting up the meter


When you power on the meter for the very first time, you will automatically be guided through the **Language** and **Time & Date** setup.

- 1 Press the Enter button  to power the meter on.

The meter automatically displays the **Language** screen. A green dot indicates which language is currently active.

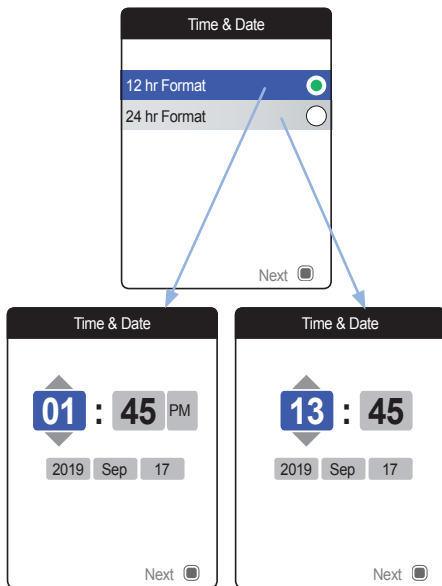- 2 Use the up/down arrows button ▲▼ to select your preferred language. The newly selected language is highlighted (blue).
- 3 Press the Enter button to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button again to save the selected language.



The meter now displays the screen for setting the time format.


- 5 Use the up/down arrows button ▲▼ to select the time format for the display (24 hour or 12 hour am/pm). The newly selected format is highlighted (blue).
- 6 Press the Enter button to confirm the selection, which is now indicated by the green dot.
- 7 Press the Enter button again to save the setting. The screen for setting the time and date follows next.

All test results are saved together with time and date. Analyzing stored test results or using reminders will only work with time and date correctly set. Always check (and, if necessary adjust) the time and date after replacing the batteries.

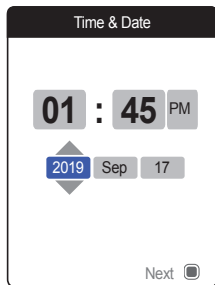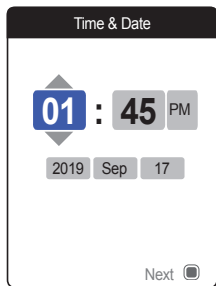Throughout this manual the 24 hr format is used on example screens.

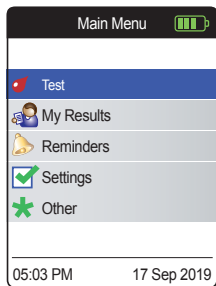- 8 Use the up/down arrows button ▲▼ to set the current hour.
- 9 Press the Enter button to confirm the hour. The next input field (minutes) will be highlighted.
- 10 Proceed with setting the remaining fields (year, month, day). The Enter button always confirms a setting (and highlights the subsequent field, if available), the Back button  allows you to jump back to the previous field.

The initial setup is now finished, and the meter displays the **Main Menu**. For a complete overview of setup options (including those which are not part of the initial setup) see the chapter *Meter settings* starting on page 87.

If you wish to power off the meter after setup:

- 11** Press and hold the Enter button  until the meter powers off.

Code chip

The code chip provides the meter with important information which it needs to perform the coagulation test. The chip contains information about the test method, the lot number and the expiry date. The meter is ready to use once the code chip has been inserted.

If you use a new test strip lot and have not inserted the corresponding code chip yet, you must do so at latest after inserting the test strip. Otherwise you cannot perform a test. The meter displays the number of the code chip belonging to the new test strip lot.

Once the code chip information has been read by the meter, it will be stored for future use. You may remove the code chip at this point. Up to 5 code chip data sets can be stored in the meter simultaneously.

Protect the code chip from moisture and equipment which produces magnetic fields, e.g. loudspeakers, induction stoves or other electronic devices.

Inserting the code chip

- 1 Remove the old code chip if one is inserted in the meter. Dispose of the old code chip with your household waste.
- 2 Slide the new code chip into the slot on the side of the meter as shown until you feel it snap into place.

This page intentionally left blank.

Testing a Capillary Blood Sample

What you need

- CoaguChek Vantus meter
- The code chip already inserted in the meter (each test strip container has its own code chip)
- Test strips belonging to the code chip mentioned above
- Lancing device (e.g. CoaguChek XS Softclix)
- Lancet (e.g. CoaguChek Softclix Lancet)
- Lint-free cloth or paper tissue

Important notes

Always ...

- close the container immediately after removing a test strip.
- operate the meter at an ambient temperature between 59 °F and 90 °F.
- place the meter on a level, stable surface (table) or hold it so that it is roughly horizontal.
- follow the information on correct handling of test strips in the package insert.
- keep the test strip guide and housing clean. Refer to the chapter *Cleaning and Disinfecting the Meter and the Lancing Device*.

Never ...

- store the meter at extreme temperatures.
- store the meter in damp or humid conditions without protection.
- remove or insert the code chip while the meter is performing a test.
- touch or remove the test strip during a test.
- wait more than 15 seconds after lancing the fingertip before applying the blood.
- add more blood after the test has begun.
- perform a test with a drop of blood from a previous puncture.

To avoid inaccurate results, make sure you follow the above recommendations.

Getting a good capillary blood sample

To get a suitable drop of blood:

- Wash your hands in warm water and dry thoroughly.

WARNING

Residues of water on the skin can dilute the drop of blood and produce false results.

- Let your hand hang at your side before lancing your finger.
- Immediately after lancing, massage gently along the side of your finger to obtain a sufficiently large blood drop without pressing or squeezing too hard.

Preparing to test

- 1 Have the test strip container to hand.
- 2 Make sure that the code chip belonging to these test strips is available or the information has previously been stored in the meter.
- 3 Prepare the lancing device by inserting a fresh lancet.

Do **not** lance your finger at this point. Wait until step 10 on page 48 to do so.

Performing a test

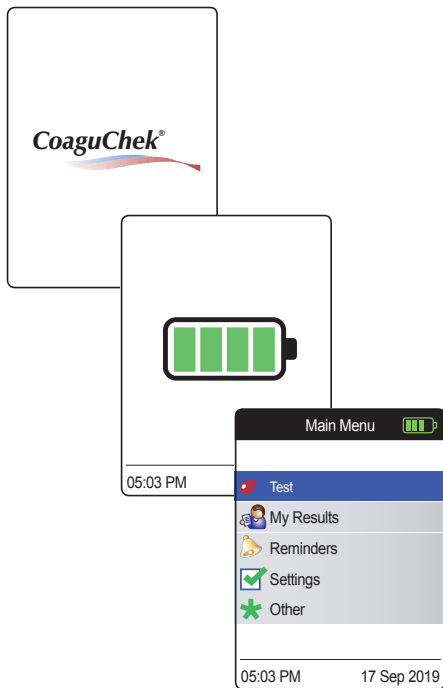
- 1 Wash your hands with soap and warm water. Dry them thoroughly.

WARNING

Residues of water on the skin can dilute the drop of blood and produce false results.

- 2 Place the meter on a level, vibration-free surface or hold it in your hand so that it is roughly horizontal.
- 3 Power the meter on by pressing the Enter button . Alternatively, you can insert a test strip to power it on, which will put the meter – after startup – into test mode directly.

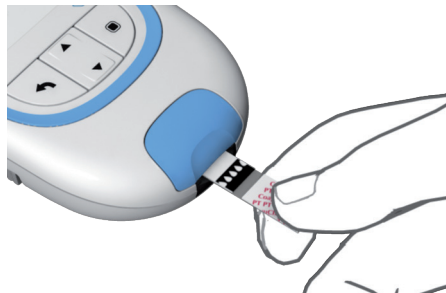
- 4 Check the battery level during startup (or in the **Main Menu** afterwards).

If the meter displays a Battery Low message, you can still perform up to 3 tests. Replace the batteries as soon as possible.

- 5 Check that the time and date are correct. If necessary, re-enter time and date (see page 91).

If you have powered on the meter by inserting a test strip, the meter will skip the following steps 6 to 9 and proceed to the steps described on page 47.

- 6 After startup, **Test** is preselected and highlighted. Press the Enter button  to start the test.

The test strip symbol prompts you to insert a test strip.

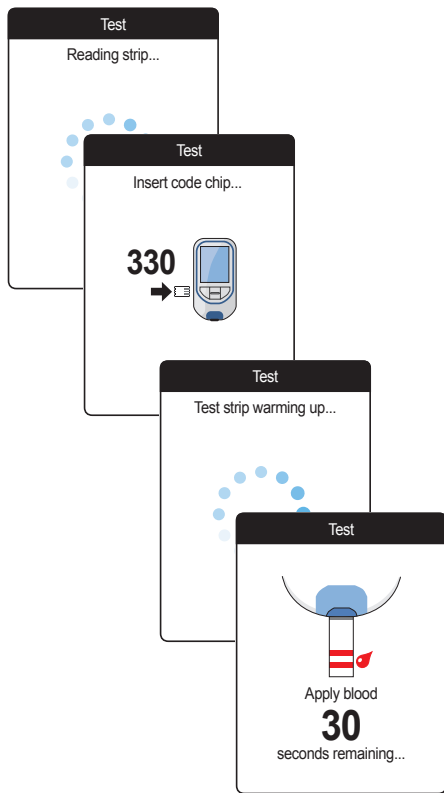
- 7 Remove a test strip from its container. Immediately after removing a test strip, close the container again with the stopper.

NOTICE

Exposure to external influences (e.g. humidity) may deteriorate the test strips and may lead to error messages.

- 8 Hold the test strip so that the lettering, the arrows and the blood drop symbols are facing upward.
- 9 Slide the test strip into the test strip guide in the direction indicated by the arrows.

Slide the test strip in as far as it will go. A beep tone indicates that the test strip has been detected (unless you have disabled the beeper function).

- The meter reads the strip information (provided by the barcode on the test strip) and checks if it matches code chip information already stored in the meter.
- If you haven't inserted the code chip yet or in case of a mismatch, you will be prompted to insert the correct code chip.
- When the test strip check has been completed successfully, the meter indicates that the test strip is warming up.
- Once the warming-up process is complete, the meter displays the test strip and blood drop symbols. These symbols and a beep tone (unless you have disabled the beeper function) indicate that the meter is ready to perform the test and is waiting for you to apply blood.

At the same time, a 180-second countdown begins. You must apply the drop of blood to the test strip within this time, otherwise you will receive an error message.

10 Now lance the side of a fingertip with the lancing device.

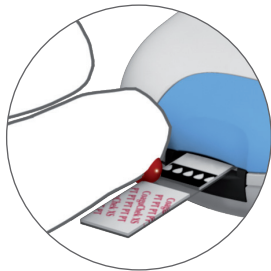
We recommend obtaining the capillary blood from the side of the fingertip as this causes the least pain.

11 Massage the lanced finger until a drop of blood is formed.

⚠ CAUTION

Do not press or squeeze the finger.

12 Apply the **first drop of blood** from the finger to the test strip.

- 13** Apply the blood directly from the finger by touching the blood drop against the side of the semicircular, transparent sample application area of the test strip.

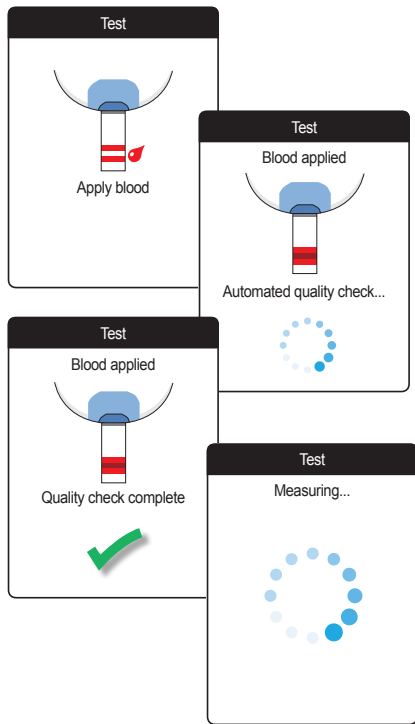
To prevent excess blood from reaching the meter, apply blood via side-dosing directly from fingertip.

Alternatively, you can apply the blood directly from the finger to the center of the semicircular, transparent sample application area of the test strip. The test strip draws up the blood by capillary action.

During this process you must hold the blood drop to the test strip until the next screen indicates that the blood has been applied successfully.

⚠ CAUTION


Apply the blood drop to the test strip **within 15 seconds** of lancing the fingertip. Applying blood after this period of time could falsify the result, as the coagulation process could already have begun.

A beep tone indicates that you have applied enough blood (unless you have disabled the beeper function). The blood drop symbol disappears and the test starts.

Do not add more blood. Do not touch the test strip until the result is displayed.

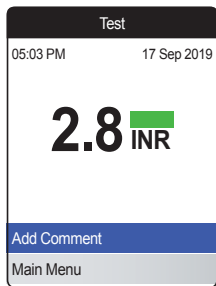The meter now performs an automatic quality control check on the test strip. Following a successful outcome of the quality control check, the coagulation measurement starts. The circular progress bar spins until the meter has determined the result.

The result is displayed and automatically saved to memory.


When interpreting your result, please also read the test strip package insert carefully.

If a blue control bottle icon  is displayed along with the result:


This may occur if the hematocrit value is very low or due to erroneous blood collection (e.g., wet hands). Make sure your hands are dry. Repeat the test. If the icon is displayed again, please contact your healthcare professional or seek immediate medical attention.

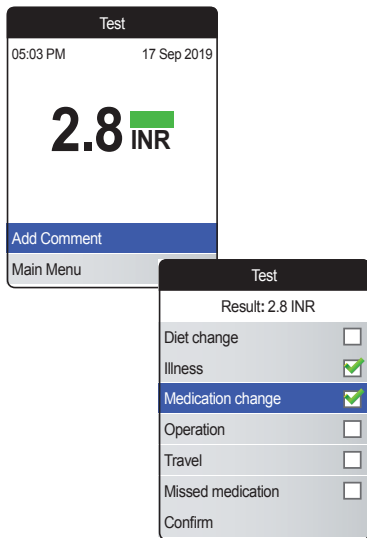If you wish, you can add up to six predefined comments to a test result. Comments can, for example, provide additional information about the test conditions or your general state of health.

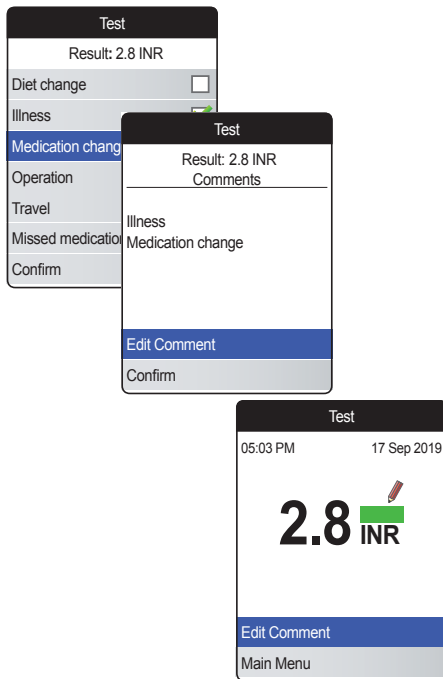These comments can **only** be added immediately after the test result is displayed (before the meter is powered off); later editing is not possible.

- 14** To add a comment, use the up/down arrows button ▲▼ to highlight **Add Comment**.
- 15** Press the Enter button  to proceed to the comment list.
- 16** Use the up/down arrows button ▲▼ to highlight the comment(s) you wish to add.
- 17** Press the Enter button  to confirm the currently highlighted comment. A green checkmark ✓ appears behind each selected comment.
- 18** Once you have selected the desired comment(s), use the up/down arrows button ▲▼ to highlight **Confirm**.
- 19** Press the Enter button  to proceed to the next step.

20 The next screen displays a list of all previously selected comments. Check if the list is correct, and use the up/down arrows button ▲▼ to highlight **Confirm**.

21 Press the Enter button  to save the selected comments with the test result and exit this menu.


The result screen now shows the Comment symbol . The test is completed.

- 22 Remove the test strip from the test strip guide.
- 23 Press and hold the Enter button  until the meter powers off.
- 24 Dispose of the used test strip with your regular household waste.

**WARNING**

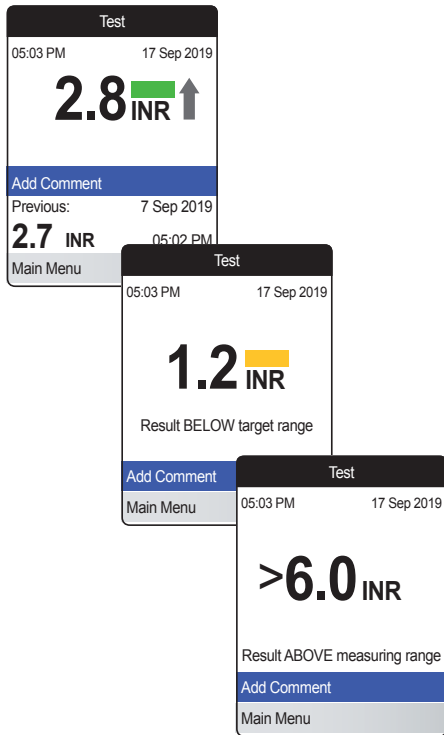Dispose of used lancets carefully (e.g. use a sturdy sharps container with lid) so that the needles cannot cause injury to yourself or to others.

Notes concerning the results display

If you have activated the detailed result format (as shown in the first screen on the left), values that are above the previously measured test result are flagged with an up arrow. Results that are below the previously measured test result are flagged with a down arrow.

- All values that lie within the set target range are marked with a green bar.
- Values above or below the target range are marked with a blue (above) or yellow (below) bar plus a corresponding explanatory text.
- Values above or below the measuring range of the meter are marked with a > (above) or < (below) symbol plus a corresponding explanatory text.

For further information about setting the result format or the target range, see *Meter settings* starting on page 87.

The **target range** is the range that you have agreed with your Healthcare Professional. Your test result should fall within the target range.

The **measuring range** refers to the measuring range of the system itself and is defined by the technical limits of the meter and test strip.

The valid measuring ranges for CoaguChek XS PT Test test strips with the CoaguChek Vantus system are:

- INR: 0.8–6.0

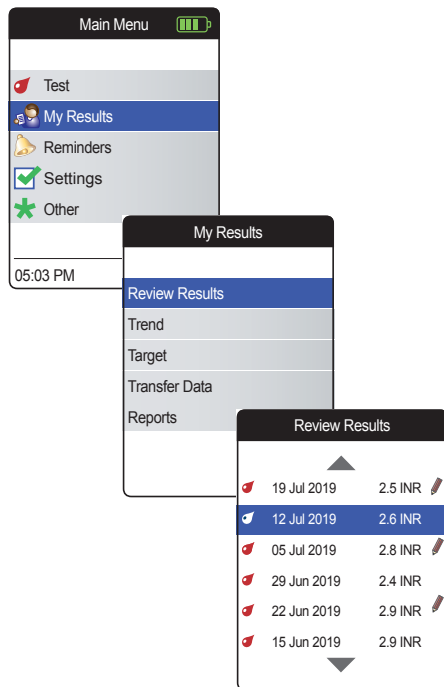
If results outside the measuring range are displayed, refer to the section *Limitations of procedure - Information for you and your physician* in the package insert supplied with the test strips, and repeat the test.

Reviewing and Handling Data

The CoaguChek Vantus meter lets you store, review, and manage test results in several ways:


- Store and review up to 400 test results including additional information and comments.
- Display result reports as graphics or tables directly on the meter.
- Transfer data to a computer for immediate evaluation, printout, and export (embedded reports).
- Transfer data to a dedicated USB or *Bluetooth* enabled device for medical communication, e.g., to your physician or healthcare provider (data transfer).

Reviewing test results

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button to select **My Results**.
- 3 Press the Enter button to open the **My Results** menu.
- 4 Select **Review Results** and press the Enter button to display the results list.

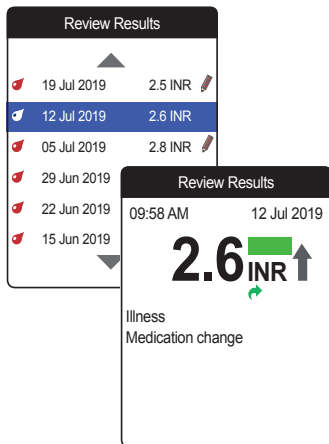All stored test results are displayed in a list with the most recent at the top.

- 5 Use the up/down arrows button ▲▼ to scroll through the list.
- 6 Press the Enter button  to open the details screen for a stored test result.

The test result will be displayed, along with the comments (entered at the time the test was performed)

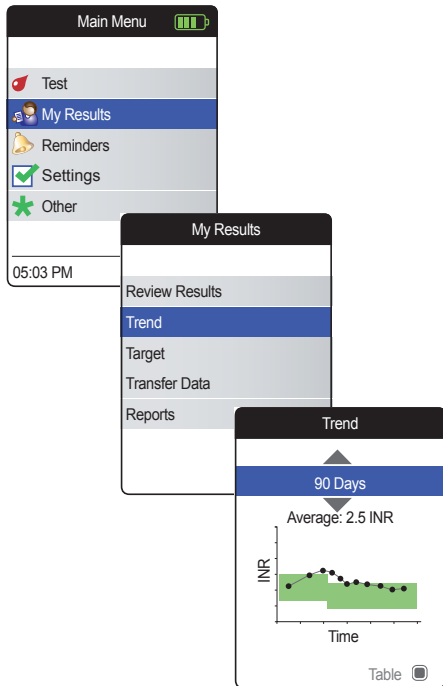
- 7 Use the Back button  to return to the results list.

Displaying reports on the meter

The CoaguChek Vantus meter offers two report types, which can be displayed directly on the meter:


- **Trend:**
a graphical or tabular report allowing a linear, time-based analysis of stored test results.
- **Target:**
a graphical or tabular report allowing a statistical analysis of stored test results.

Displaying a Trend report

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button  to select **My Results**.
- 3 Press the Enter button  to open the **My Results** menu.
- 4 Select **Trend** and press the Enter button  to display the report.

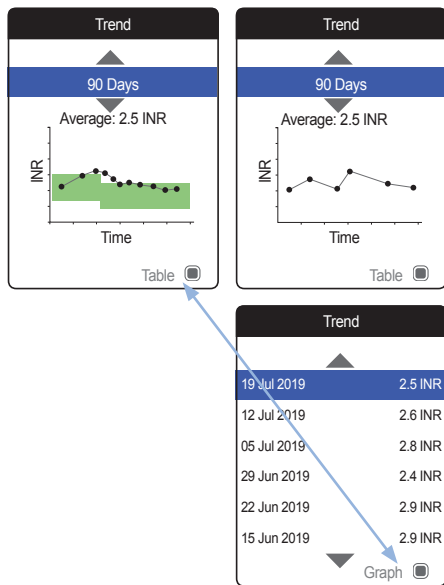In the **Trend** menu, you have the option of displaying an overview of your test results either as a list or as a graph. All test results stored for the period of time you select will be displayed in the graph as linear values (graph view) or as a list (table view).

Graph view:



Each test result (represented by the ● symbol) is plotted along the time axis according to the time when the test was performed. All test results are connected by a line.

The green shading behind the line shows the INR target range valid at that point in time (if set, otherwise not displayed).

Table view:

The test results are shown in a list. The oldest date is shown at the bottom and the most recent date at the top.

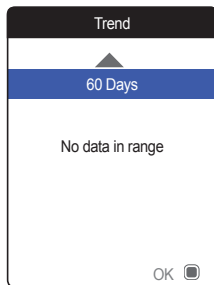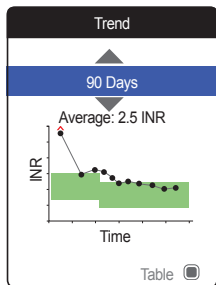
- 5 Use the up/down arrows button ▲▼ to select the time period you want. Starting from your most recent test result, you can choose a time period going back 60, 90, 120, or 180 days.
- 6 Press the Enter button to toggle between graph and table display.
- 7 Press the Back button ⬅ to return to the **My Results** menu.

Trend

19 Jul 2019	2.5 INR
12 Jul 2019	2.6 INR
05 Jul 2019	2.8 INR
29 Jun 2019	2.4 INR
22 Jun 2019	2.9 INR
15 Jun 2019	2.9 INR

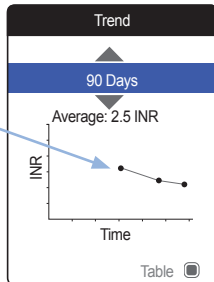Graph

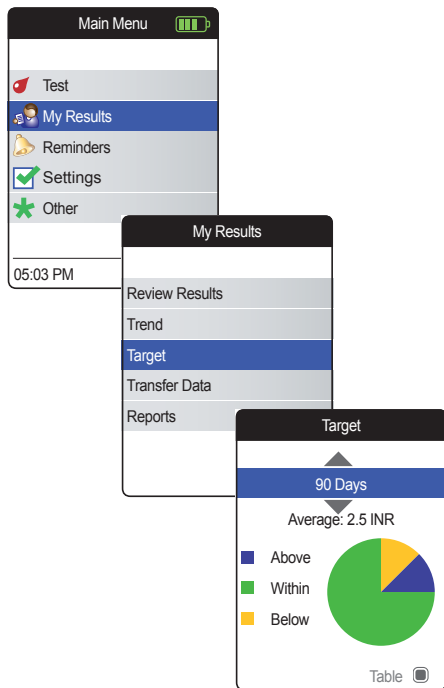
Test results above  or below  the measuring range are marked accordingly. If the selected period of time contains no test results, no graph will be displayed.

Due to an incorrect time setting, a test result may be stored with a date in the future. The example on the lower left shows a test performed after the date was inadvertently set to the next year, and setting the date back to the correct year afterwards.



These test results appear in red in the table and are not displayed in the trend graph. As there is no continuity between the test results before and after this test, all previous results (including and after this test) will not be displayed in the trend curve.

Displaying a Target report

The **Target** report is only available if you have set and enabled an INR target range (see “*Set Targets*” on page 94).

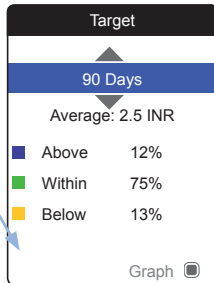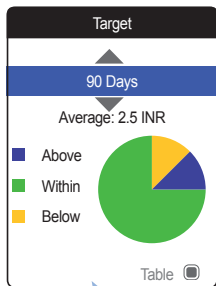
- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button  to select **My Results**.
- 3 Press the Enter button  to open the **My Results** menu.
- 4 Select **Target** and press the Enter button  to display the report.

The percentage of values above, within and below the set target range will be displayed as a pie chart. The average (numerical) value in the selected time period is also displayed. If the selected period of time contains no test results, no graph will be displayed.

- 5 Use the up/down arrows button ▲▼ to select the time period you want to evaluate. Starting from your most recent test result, you can choose a time period going back 60, 90, 120, or 180 days.
- 6 Press the Enter button to toggle between graphic and tabular display.
- 7 Press the Back button ↶ to return to the **My Results** menu.

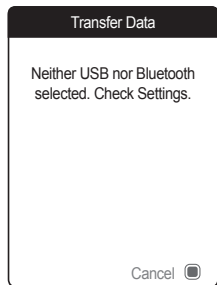Transferring data

The CoaguChek Vantus meter can communicate with a computer and/or other devices either via USB or a wireless *Bluetooth* connection. You can use this feature for the following purposes:

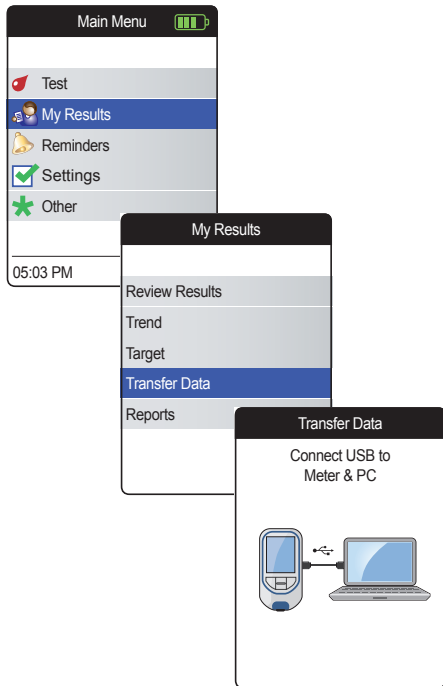
- further evaluation of stored test results in suitable computer applications
- data archiving

The preferred method (USB/*Bluetooth*) used for data transfer has to be selected in the meter settings before transferring data for the first time. Otherwise, a corresponding message will be displayed.

Both methods will be described separately on the following pages. If you are using a *Bluetooth* connection, pairing has to be completed during setup before you can perform a data transfer (see “*Data Transfer*” on page 100).

It is not possible to perform a test while a USB cable is plugged in.

Data transfer via USB

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button  to select **My Results**.
- 3 Press the Enter button  to open the **My Results** menu.
- 4 Select **Transfer Data** and press the Enter button  to start the data transfer mode.

The meter now waits for a connection to be established.

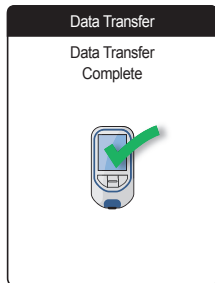
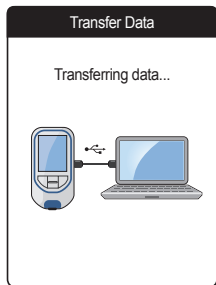
To connect the meter to a computer, you will need a suitable USB cable (included in the kit). Connect the micro USB type B plug to the meter (the USB port is located on the right-hand side of the meter) and the USB type A plug to the computer. In order to transfer data, always connect the cable to the meter first and then to the computer.


Data transfer must be actively triggered by the computer application or device app used for reading the stored data.

When the connection has been established and the download command has been received, the meter informs you about the progress.

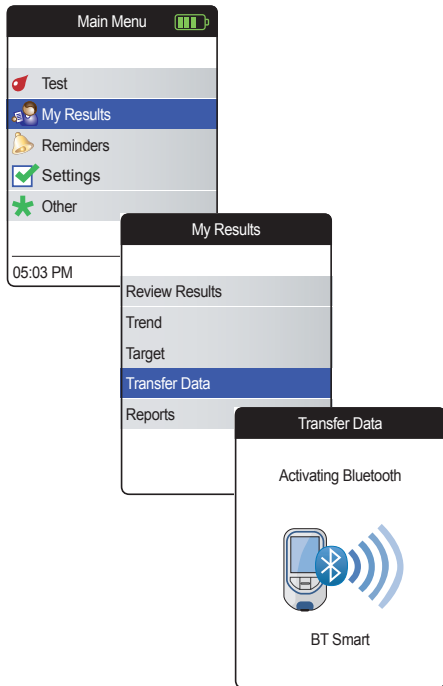To save battery power, remove the USB cable after transferring the test results to the PC.


Data transfer via *Bluetooth*

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button  to select **My Results**.
- 3 Press the Enter button  to open the **My Results** menu.
- 4 Select **Transfer Data** and press the Enter button  to start the data transfer mode.

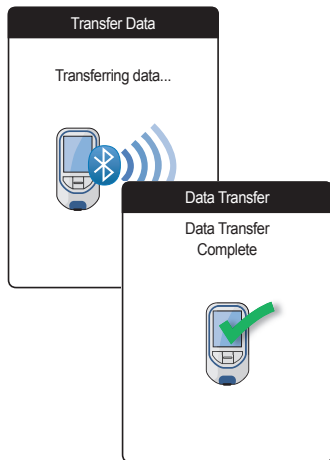
The meter now waits for a connection to the paired device to be established.

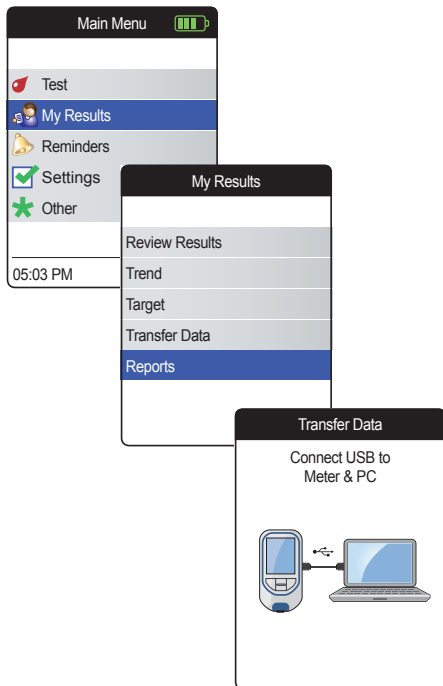When the connection has been established and the download command has been sent, the meter informs you about the progress.


Depending on the software you are using, an information screen may appear after the data transfer is complete. It may contain, e.g., information about available software updates or a change of settings applied to the meter.

Displaying reports on a computer

The reports displayed on the meter (see page 60) can also be displayed on a computer. Opening these reports on an Internet browser provides additional advantages like more detailed information, the option to print, and the ability to store results to file for further evaluation. All report data is stored on and retrieved from the meter. Reports can only be provided via a USB connection. For displaying reports:

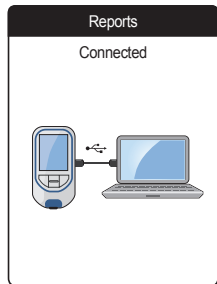- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Use the up/down arrows button  to select **My Results**.
- 3 Press the Enter button  to open the **My Results** menu.
- 4 Select **Reports** and press the Enter button .

- 5 Connect the meter to the computer via USB cable.

The display shows “Connected” when the connection has been established.

- 6 Open a window on your computer that displays the contents of the meter (as a removable disk drive).
- 7 Double click on **START.HTM** to open the file in your standard Internet browser.
- 8 After you have finished using the embedded reports, unplug the USB cable from the meter and the computer.
- 9 Your Internet browser may store local copies of the data that is visualized with the help of the embedded reports feature. Therefore, we recommend to empty the Internet browser cache every time after using this feature. In most cases, this will erase any traces of locally stored data.

When the meter is connected to a computer, it will **not** be powered via the USB cable and will **not** power off automatically. To save battery power always remove the USB cable when the connection is no longer in use.

Security settings in the Internet browser

The settings of your Internet Browser may affect your use of reports.

If you choose the security settings appropriately (e.g. “Allow active content to run files on My Computer” or similar), you can work with reports without any restrictions.

We recommend to empty your browser cache every time after using the embedded reports feature. In most cases, this will erase any traces of data that your Internet browser may have stored locally without your knowledge.

Virus scanner

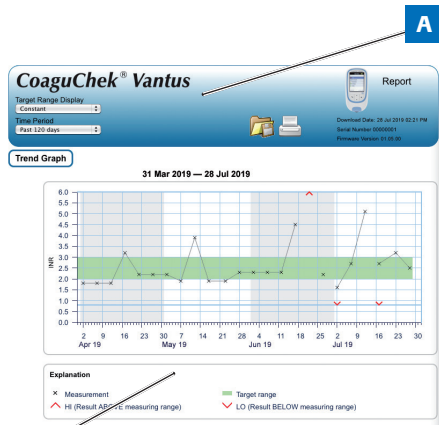
WARNING

Make sure the computer system you are using is free of malware (e.g. virus, trojan). If using a compromised computer system, the results displayed can be incomplete or falsified.

Security software (e.g. virus scanners) may advise you to be careful with connected external devices of unknown origin. However, the CoaguChek Vantus meter can be connected to your PC.

Reports overview

A report displayed in your Internet browser contains the following elements:

A Report Header

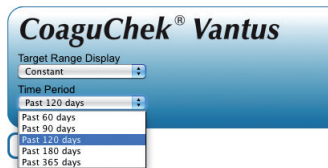
Displays information about the meter (like serial number, software version etc.), allows you to set the **Time Period** for statistics and the **Target Range Display**, and contains buttons for printing and saving the report to file.

B Report Content

The page body contains the **Trend Graph**, the **Target Graph**, and a chronological list of **Results**.


Language and your personal target range will be displayed as set in the meter.

Time period

Starting from your most recent test result, you can choose a **Time Period** going back 60, 90, 120, 180, or 365 days. This setting is applied to both the Trend and Target graph.

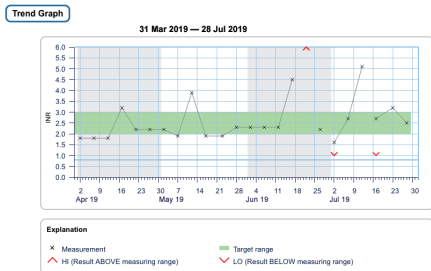Trend graph

All test results stored within a selected period of time will be displayed in the chart as linear values. Each test result (represented by the **x** symbol) is plotted along the time axis according to the time when the test was performed. Test results above **▲** or below **▼** the measuring range are marked accordingly. All test results are connected by a line in chronological order.

Vertical lines on the Trend graph mark the beginning of each week.

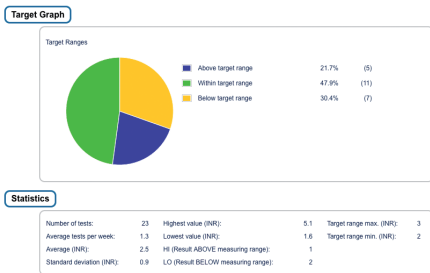
Target range display

The green shading behind the line shows the INR target range (if set, otherwise not displayed). You may set the **Target Range Display** to:

- **Constant:** the current target range set in the meter is applied to all results in the selected time period.
- **Individual:** the green bar shows the target range valid at that point in time.

Target graph

The percentage of values above, within and below the set target range will be displayed as a pie chart. The average (numerical) value and the total number of results in that range are also displayed.

Below the Target graph you will find further statistical information calculated from all values within the selected time period.

List of results

Results

31 Mar 2019 — 28 Jul 2019				
Date and Time	PT Coagulation (INR)	Target Range (INR)	Comments	Transferred
28 Jul 2019 03:14AM	2.5	2.0 - 3.0		
23 Jul 2019 01:12AM	3.2	2.0 - 3.0 Above target range		
17 Jul 2019 08:12PM	2.7	2.0 - 3.0	Missed medication	
08:10PM	LO	2.0 - 3.0 Below target range		
12 Jul 2019 09:07PM	5.1	2.0 - 3.0 Above target range	Illness, Medication change, Operation	
07 Jul 2019 07:05PM	2.7	2.0 - 3.0		
02 Jul 2019 05:05PM	1.6	2.0 - 3.0 Below target range	Missed medication	

All test results are listed in chronological order, with the latest result on top. The list contains the following columns:

- Date and Time**
- PT Coagulation**
 The result value is shown. If the result is outside the measuring range, either LOW or HIGH will be shown.
- Target range**
 The target range valid at that point in time. If the result is outside the target range the cell is colored blue (above) or yellow (below).
- Comments**
- Transferred**
 The upload icon indicates that a result has previously been transferred using the “Transfer data” function of the meter. Viewing a report on the computer as described here does not constitute data transfer and does not activate this indicator.

Printing reports

All reports displayed on the screen can also be printed out immediately.

In order to achieve the best results, use the print button  in the report header. Do not use the browser's integrated print function.

- 1 Click the print button  in the report header.
- 2 Select the desired printer from the print dialog box, and adjust print parameters as needed, depending on the available options.
- 3 Start the printing process.

Using data in external applications

For evaluating test results using other software, you can save the data as a ***.csv** file. The file extension CSV stands for **C**omma **S**eparated **V**alues. These files can be used by various applications, for example a text editor or a spreadsheet application.

The *.csv file always contains **all** the test results currently stored in your meter, including test results that have already been transferred during previous data transfers.

- 1 Click the Save file button  in the report header.
- 2 Select the desired save options in the following dialog box.

The saved file is handled by the system like a download performed by the Internet browser, and the corresponding settings apply. This will, for example, determine if the file is merely stored or directly opened in an application. If saving the *.csv file does not work as expected check the relevant system and browser settings.

The saved *.csv file contains the following information:

- Serial number of the meter
- Date and time of data download to the computer (for creating this *.csv file)
- Date, time and value of each stored test result

Reminders and Meter Settings

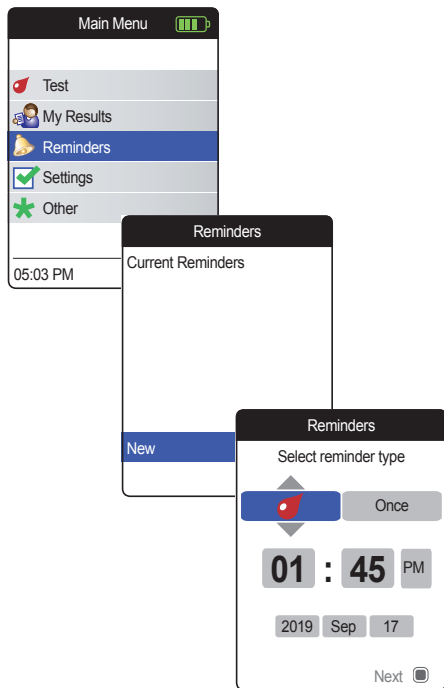
Setting up reminders

The CoaguChek Vantus meter lets you set reminders that work just like an alarm clock telling you it's time for an action. Beyond the abilities of an alarm clock, these reminders also tell you **which** action you have scheduled at that point in time. You may set four different kinds of reminders:

- Test  reminders
- Medication  reminders
- Appointment  reminders
- Other  reminders

Each reminder can be scheduled as a single reminder, or as a recurring reminder (daily, or in 1-4 week intervals).

New reminder


To set a new reminder:

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Select **Reminders** and press the Enter button to open the corresponding menu.
- 3 Select **New** and press the Enter button to set up a new reminder.


The **Reminders** screen opens and displays the options you can set:

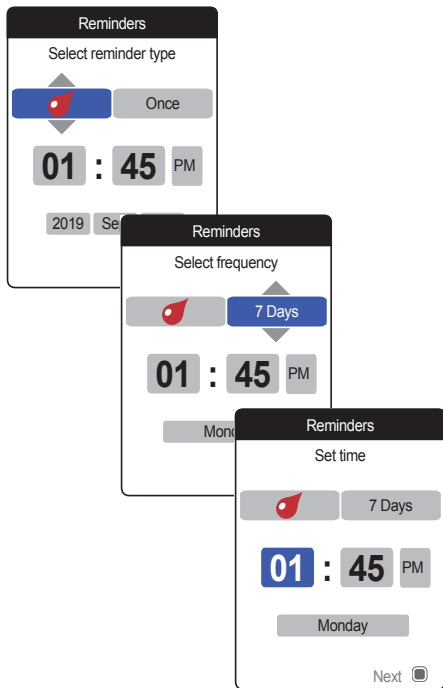- Type (default: (Test))
- Interval (default: once)
- Time and date (default: current time on the next day)

- 4 Use the up/down arrows button ▲▼ to set the desired type (test, medication, appointment, other).
- 5 Press the Enter button to confirm the selected type. The next reminder parameter (frequency) will be highlighted.
- 6 Use the up/down arrows button ▲▼ to set the desired interval (once, daily, every 7, 14, 21, or 28 days).
- 7 Press the Enter button to confirm the selected interval. The next reminder parameter (hour) will be highlighted.
- 8 Set the remaining time/date parameters, always using the up/down arrows button ▲▼ to set the parameters, and the Enter button to confirm and jump to the next parameter.

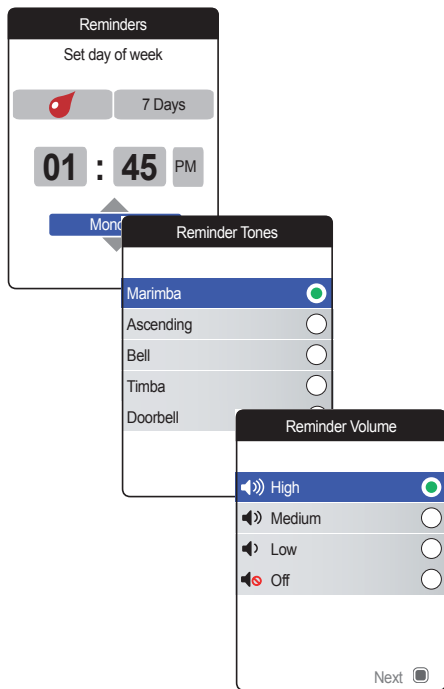Depending on the chosen interval, there will be different time/date parameters to set.


After entering and confirming the day, you are prompted to set the audio signal for the reminder.

9 Use the up/down arrows button ▲▼ to select the desired **Reminder Tone**, which may, for example, help you to distinguish acoustically between reminder types. Press the Enter button to confirm your selection. A green dot indicates the selected tone.

10 Press the Enter button again to save the selection and proceed.

11 Select the desired **Reminder Volume** and press the Enter button to confirm your selection. A green dot indicates the selected volume.

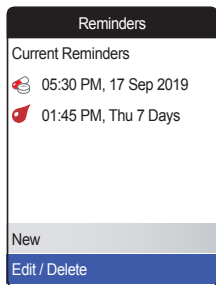
12 Press the Enter button to save the reminder.

The newly set reminder is now shown in the list of reminders. You can now

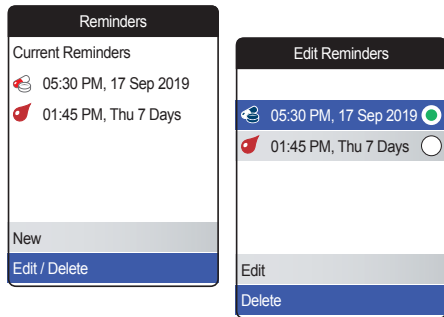- set another **New** reminder
- **Edit/Delete** the existing reminder
- exit the **Reminders** menu with the Back button  and return to the Main Menu.

Editing or deleting a reminder

To edit or delete a reminder:

- 1 Select **Edit/Delete** and press the Enter button  to proceed.
- 2 Select the desired reminder and press the Enter button  to confirm your selection. A green dot indicates the selected reminder.
- 3 Select **Edit** or **Delete** and press the Enter button  to proceed.

If you have chosen to edit the reminder, proceed as described in section “*New reminder*”. If you have chosen to delete the reminder, it will be deleted after confirmation.

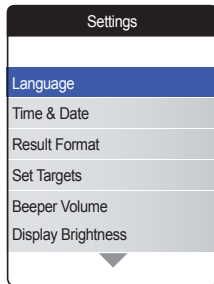
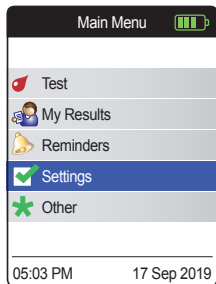
The **Edit/Delete** button is only available if there is at least one reminder listed. The **New** button is only available if there are fewer than four reminders listed.

Meter settings

The initial meter setup was completed after you powered on the meter for the first time. Use the following descriptions to set options that are not covered by the initial setup (e.g., the target range) or to edit settings already made.

Always perform the following two steps before selecting one of the options in the **Settings** menu:

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Select **Settings** and press the Enter button  to open the corresponding menu.

Settings overview


Setting	Options
Language	<ul style="list-style-type: none">English*Español
Time & Date	Time format <ul style="list-style-type: none">24-hour format (24 hr)12-hour format (12 hr) with a.m./p.m.*
Result Format	<ul style="list-style-type: none">Simple*Detailed
Set Targets	Time period (statistics) <ul style="list-style-type: none">On<ul style="list-style-type: none">High valueLow valueOff*60 days*90 days120 days180 days

* Default settings are marked with an asterisk.

Setting	Options
Beeper Volume	<ul style="list-style-type: none">▪ High*▪ Medium▪ Low▪ Off
Display Brightness	<ul style="list-style-type: none">▪ High*▪ Medium▪ Low
Button Click	<ul style="list-style-type: none">▪ On*▪ Off
Data Transfer	<ul style="list-style-type: none">▪ None*▪ Bluetooth▪ USB

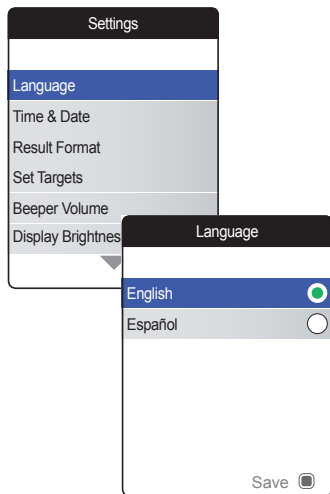
* Default settings are marked with an asterisk.

Language

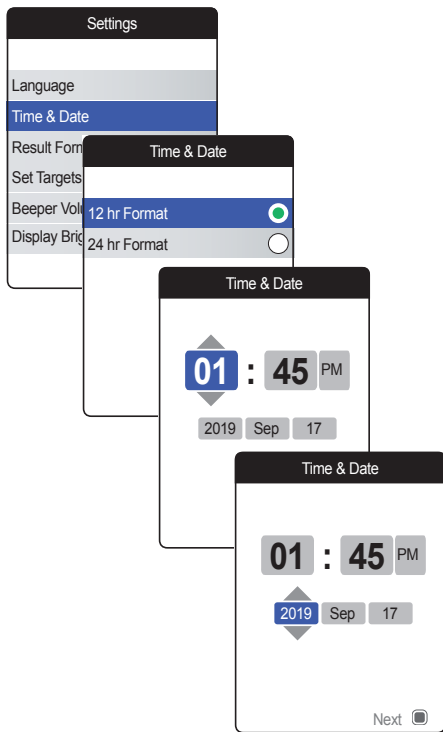- 1 Select **Language** and press the Enter button to open the corresponding menu.
- 2 Use the up/down arrows button to select your preferred language. The newly selected language is highlighted (blue).
- 3 Press the Enter button to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button again to save the selected language.
- 5 On the following confirmation screen select **Confirm** and press the Enter button to switch the meter to the selected language.

Time & Date

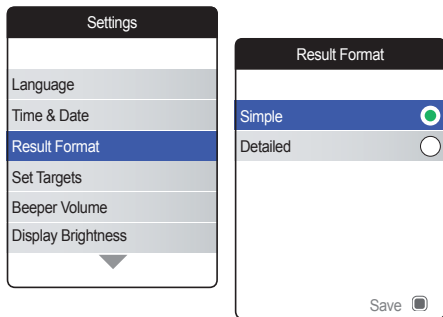
- 1 Select **Time & Date** and press the Enter button  to open the corresponding menu.
- 2 Use the up/down arrows button  to select the time format for the display (24 hour or 12 hour am/pm).
- 3 Press the Enter button  to confirm and save the setting. The screen for setting the time and date follows next.
- 4 Use the up/down arrows button  to set the current hour.
- 5 Press the Enter button  to confirm the hour. The next input field (minutes) will be highlighted.
- 6 Proceed with setting the remaining fields (year, month, day). The Enter button  always confirms a setting (and highlights the subsequent field, if available). The Back button  allows you to jump back to the previous field.

In order to ensure a correct time and date setting, the meter checks the date for plausibility. The meter will display a message in the following cases:

- the date you want to set precedes the date of the most recently stored measurement
- the date you want to set is six months later than the last stored measurement result

Result Format

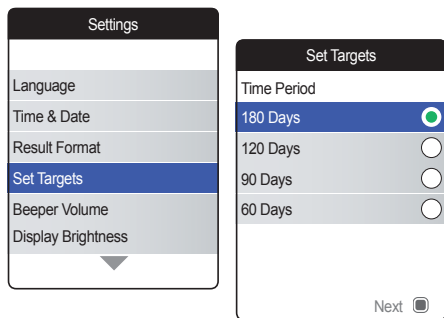
Choose **Detailed** if you want to see your current result compared with your previous result on the same screen. An up arrow indicates that your current result is higher, and a down arrow indicates that it is lower than your previous result (see page 55).

- 1 Select **Result Format** and press the Enter button  to open the corresponding menu.

The currently active result format is indicated by a green dot on the right.

- 2 Use the up/down arrows button  to select your preferred result format. The newly selected format is highlighted (blue).
- 3 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button  again to save the selected result format.

Set Targets

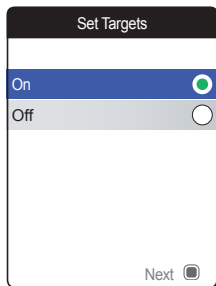
A target range defines the minimum and maximum values which should be achieved during therapy. Reports and statistics can then include information about values within, above or below this target range.

- 1 Select **Set Targets** and press the Enter button  to open the corresponding menu.

The currently selected time period, which will be used as the default time period for reports and statistics, is indicated by a green dot on the right.


- 2 Select the desired time period.
- 3 Press the Enter button  to confirm the selected time period, which is now indicated by the green dot.
- 4 Press the Enter button  again to proceed to the next step.

The current activation status is indicated by a green dot on the right.

A newly set or edited target range is only valid for tests performed **after** this setting is made, and will not affect test results already stored in memory.

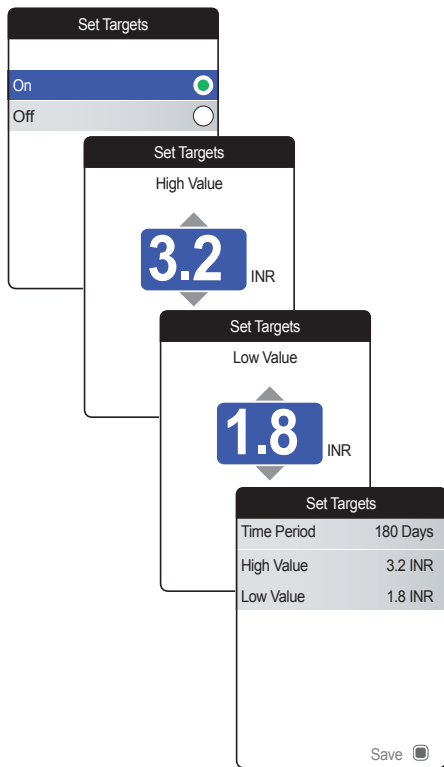
- 5 Use the up/down arrows button ▲▼ to select **On** or **Off**.
- 6 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 7 Press the Enter button  again to proceed to the next step.

If you have selected the option **Off**, the **Set Targets** setup ends here. If you have selected the option **On**, proceed as follows:

- 8 Use the up/down arrows button ▲▼ to set the desired high value.
- 9 Press the Enter button to save the set value.
- 10 Repeat the last steps to set the low value.
- 11 Press the Enter button to save the settings.

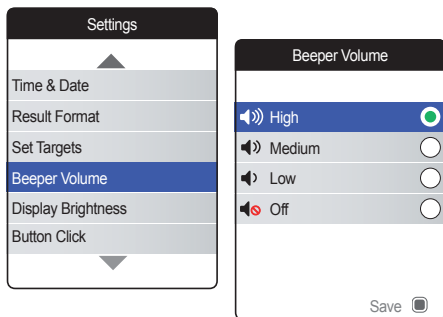Beeper Volume

If the beeper is activated, the meter always beeps when an action is taken, for example a test strip is inserted or a sufficient sample has been applied.

The beeper volume can be set or deactivated. However, in case of an error the meter will always beep for safety reasons, even if you have set the beeper function to **Off**. To set the beeper volume:

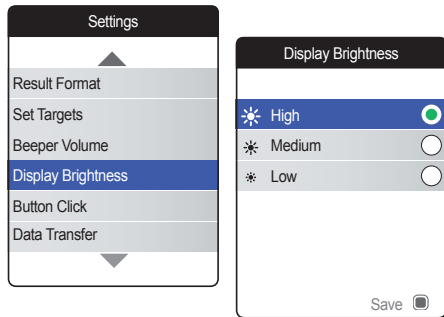
- 1 Select **Beeper Volume** and press the Enter button  to open the corresponding menu.

The currently set volume is indicated by a

right.

- 2 Use the up/down arrows button  to select the desired volume.
- 3 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button  again to save the selected volume.

Display Brightness

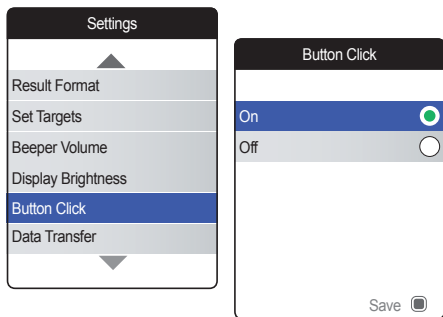
- 1 Select **Display Brightness** and press the Enter button  to open the corresponding menu.

The currently set display brightness is indicated by a green dot on the right.

- 2 Use the up/down arrows button  to select the desired display brightness. The display will temporarily switch to the currently highlighted brightness.
- 3 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button  again to save the selected display brightness.

To improve battery lifetime, choose a lower display brightness setting.

Button Click

The meter can be set to emit a click sound whenever you press a button.

To set the button click:

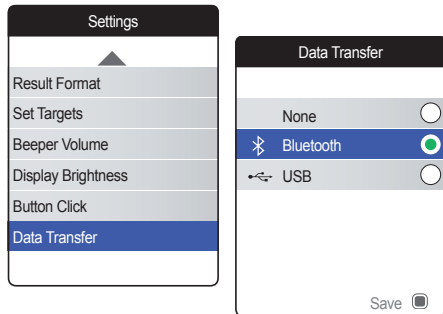
- 1 Select **Button Click** and press the Enter button  to open the corresponding menu.

The current status is indicated by a green dot on the right.

- 2 Use the up/down arrows button  to select **On** or **Off**.
- 3 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button  again to save the selected status.

The setting of the beeper volume also applies to the button click. If the beeper volume is set to **Off**, the button click will also be muted.

Data Transfer

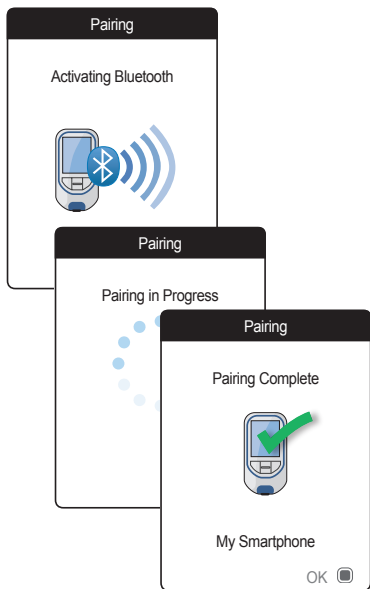
The CoaguChek Vantus meter can communicate with a computer and/or other devices via USB or a wireless *Bluetooth* connection. Using the Data Transfer option (see the chapter *Transferring data* starting on page 66) enables the transfer mode (USB or *Bluetooth*) preselected here.

To set up data transfer:

- 1 Select **Data Transfer** and press the Enter button  to open the corresponding menu.

The currently set transfer mode is indicated by a green dot on the right.

- 2 Use the up/down arrows button  to select the desired transfer mode.
- 3 Press the Enter button  to confirm the selection, which is now indicated by the green dot.
- 4 Press the Enter button  again to save the selected transfer mode.

If you have selected **None** or **USB**, the data transfer setup ends here.

If you have selected **Bluetooth**, the meter shows the screen "Activating Bluetooth".

- 5 Start the *Bluetooth* scan on the device you want to pair with and select "CoaguChek".

When a connection has been established, the meter continues the pairing process.

- 6 Press the Enter button  to confirm and finish the pairing process.

The meter can be paired via *Bluetooth* with one device at a time. This means that an existing *Bluetooth* connection has to be deleted before you can pair the meter with another device.

This page intentionally left blank.

Cleaning and Disinfecting the Meter and the Lancing Device

Overview

NOTICE

To avoid malfunction of the meter, follow the procedures below to clean and disinfect the meter and the lancing device.

- Make sure the meter is turned off.
- Do not use sprays of any sort.
- Ensure that the cloth or cotton swab/bud is only damp, not wet, to protect against moisture entering the meter.

To prevent excess blood from reaching the meter, apply blood via side-dosing directly from fingertip.

What is the difference between cleaning and disinfecting?

Cleaning is the removal of dirt from the meter or lancing device.¹

Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms (bloodborne pathogens) from the meter or lancing device.¹

When to clean and disinfect the meter and the lancing device

- Clean and disinfect the meter and/or the lancing device to remove visible dirt or other material on a regular basis.
- Always clean and disinfect the meter and/or the lancing device when there is blood on it.
- Clean and disinfect the meter and/or the lancing device before allowing anyone else to handle the meter and/or the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device.

¹ Rutala WA, Weber DJ, Healthcare Infection Control Practices Advisory Committee (HICPAC). Guideline for disinfection and sterilization in healthcare facilities, 2008. Centers for Disease Control and Prevention. Atlanta, Georgia. 2008.

Approved cleaning and disinfecting product

The following product has been approved for cleaning and disinfecting the meter and lancing device:

- Super Sani-Cloth® Germicidal Disposable Wipes (EPA¹ reg. no. 9480-4) Pre-moistened disinfecting cloths (active ingredient 0.5% quaternary ammonium chlorides and up to 60% isopropanol)

Super Sani-Cloth Germicidal Disposable Wipes can be purchased from Amazon.com, Officedepot.com and Walmart.com.

- Do not use any other cleaning or disinfecting solutions. Using solutions other than Super Sani-Cloth Germicidal Disposable Wipes could result in damage to the meter and lancing device.
- The effect of using more than one product interchangeably to clean and disinfect the meter and lancing device has not been tested. Always use Super Sani-Cloth Germicidal Disposable Wipes to clean and disinfect the meter and lancing device.

¹ Environmental Protection Agency

Cleaning the meter housing

- Roche has tested the approved product for a total of 520 cycles, which is equal to cleaning and disinfecting once per week for a 5-year period.

NOTE: For technical assistance or questions on cleaning and disinfecting, contact CoaguChek Technical Services at 1-800-428-4674.

- Ensure that the blue test strip guide cover remains tightly closed while cleaning the meter housing.
- Do not let liquid accumulate near any opening.

NOTICE

Ensure that **no** liquid enters the meter when you clean the exterior. If moisture enters the meter, it may cause malfunction of the instrument.

- 1** Wash hands thoroughly with warm soapy water.
- 2** Turn off the meter and place it on a flat surface, such as a table.
- 3** Wipe the entire meter surface with the approved cleaning and disinfecting product. Always use the same product for both cleaning and disinfecting.
- 4** Carefully wipe around the test strip slot and other openings. Make sure that no liquid enters any opening.

Cleaning the test strip guide and test strip guide cover

Squeeze off excess cleaning solution or blot on a dry paper towel to remove any excess cleaning solution before cleaning the surface of the meter.

- 1 With the meter powered off, remove the test strip guide cover to clean it. (Use your thumbnail to open the cover of the test strip guide by pressing its front edge upward.) Move the cover safely away from the meter. Then wipe it clean using Super Sani-Cloth Germicidal Disposable Wipes. Let the test strip guide cover dry **for at least 10 minutes** before re-attaching it.

- 2 Hold the meter upright with the test strip guide facing down.
 - Clean the easily accessible white areas with a Super Sani-Cloth Germicidal Disposable Wipe.
 - Ensure the Super Sani-Cloth Germicidal Disposable Wipe is only damp, not wet. Wipe away residual moisture and fluids.
 - With the test strip guide cover off and holding the meter upright with the test strip guide facing down, use a fresh Super Sani-Cloth Germicidal Disposable Wipe and fold one edge of the wipe. Clean the test strip guide by gently wiping the easily accessible white areas, making sure that no liquid pools into the test strip guide.

NOTICE

Damage to the instrument

- Ensure that no liquid enters the meter when cleaning the test strip guide. If moisture enters the meter, it may cause malfunction of the instrument.
- Do not insert any objects in the test strip guide. Doing so may damage the electrical contacts behind the test strip guide.

On completion of cleaning:

- 3 With the cover off, let the test strip guide dry for **at least 10 minutes** before re-attaching the test strip guide cover and testing again.
 - Visually verify that no residual moisture is seen anywhere on the test strip guide and cover at the completion of cleaning.
 - Ensure that the test strip guide and cover are completely dry before assembling them.

**WARNING**

- 4 Re-attach the test strip guide cover to the housing. Make sure that the cover is properly closed. You will hear it snap into place.
-

Disinfecting the meter

NOTE: Clean the meter prior to each disinfection step

Remember to keep the meter on a level surface prior to disinfecting and powered off.

Squeeze off excess disinfecting solution or blot on a dry paper towel to remove any excess disinfecting solution before disinfecting the surface of the meter.

- 1** Use a fresh Super Sani-Cloth Germicidal Disposable Wipe to disinfect by gently wiping the outside of the meter three times horizontally and three times vertically and carefully wipe around the test strip guide area, making sure that no liquid enters the test strip guide.
- 2** Allow the surface of the meter to remain damp with the recommended disinfecting solution. The recommended contact time is 2 minutes for the Super Sani-Cloth Germicidal Disposable Wipe.

NOTE: Ensure the meter surface stays moist for the entire contact time.

- 3 Dry the meter thoroughly with a dry cloth or gauze. Visually verify that no solution is seen anywhere on the meter at the completion of cleaning and disinfecting. Ensure that the meter is thoroughly dried after cleaning and disinfecting.

Disinfecting the test strip guide cover and test strip guide

NOTE: Clean the test strip guide prior to each disinfection step.

NOTICE**Damage to the instrument**

- Ensure that no liquid enters the meter when disinfecting the test strip guide. If moisture enters the meter, it may cause malfunction of the instrument.
- Do not insert any objects in the test strip guide. Doing so may damage the electrical contacts behind the test strip guide.
- Remember to keep the meter on a level surface prior to disinfecting and powered off. Squeeze off excess disinfecting solution or blot on a dry paper towel to remove any excess disinfecting solution before disinfecting the surface of the meter.

- 1 With the meter powered off, remove the test strip guide cover to disinfect it. (Use your thumbnail to open the cover of the test strip guide by pressing its front edge upward.) Move the test strip guide cover safely away from the meter.

- 2 With the test strip guide cover off and holding the meter upright with the test strip guide facing down, use a fresh Super Sani-Cloth Germicidal Disposable Wipe and fold one edge of the wipe. Disinfect the test strip guide by gently wiping the easily accessible white areas three times horizontally and three times vertically, making sure that no liquid pools into the test strip guide.
- 3 Allow the surface of the test strip guide to remain damp with the recommended disinfecting solution. The recommended contact time is 2 minutes for the Super Sani-Cloth Germicidal Disposable Wipe.

NOTE: Ensure the meter surface stays moist for the entire contact time.

-
- 4 Let the test strip guide dry for **at least 10 minutes** before re-attaching the test strip guide cover. Visually verify that no solution is seen anywhere on the test strip guide at the completion of cleaning and disinfecting. Ensure that the test strip guide and cover are thoroughly dried before re-attaching.

**WARNING**

- 5 Re-attach the test strip guide cover to the housing. Make sure that the cover is properly closed. You will hear it snap into place.
-

Cleaning and Disinfecting the Lancing Device

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

When to clean and disinfect the lancing device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device.

NOTE

- Do not throw away the cap after each use. Use the approved cleaning and disinfecting product on the cap.
- Remove the lancet before cleaning and disinfecting the lancing device.
- If you notice any residue around the buttons after cleaning and disinfecting the lancing device, contact CoaguChek Technical Services at 1-800-428-4674.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the lancing device.

What to clean and disinfect

Clean and disinfect the following parts of the lancing device:

- The entire lancing device surface
- The cap

How to clean and disinfect the lancing device

WARNING

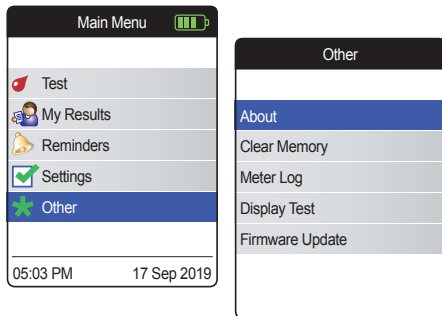
Failure to follow these instructions may damage the lancing device and stop it from working properly.

- DO NOT get any moisture into any openings.
 - Always use the same product for both cleaning and disinfecting.
-

- 1** Wash hands thoroughly with soap and water.
- 2** Wipe the entire surface of the lancing device and the inside of the cap with an approved cleaning and disinfecting product. Make sure that no liquid enters any opening.
- 3** Remove and wipe the inside of the cap
- 4** Dry the lancing device and cap with a soft cloth or gauze. Make sure no solution is seen in any opening.
- 5** For disinfecting the lancing device, get a new wipe or cloth. Repeat steps 2 and 3 for disinfecting.

Other Functions and Troubleshooting

Other functions

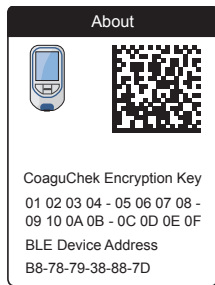
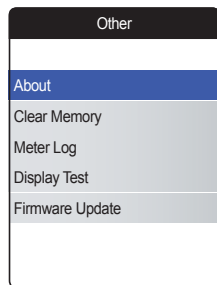
The **Other** menu of the meter contains functions relevant for service and troubleshooting purposes:

- About (meter-specific information)
- Clear Memory (deleting data and settings)
- Meter Log
- Display Test
- Firmware Update

Always perform the following two steps before selecting one of the options in the **Other** menu:

- 1 Power the meter on by pressing the Enter button . Wait until the **Main Menu** is displayed.
- 2 Select **Other** and press the Enter button  to open the corresponding menu.

About (manufacturing information and data encryption key)

In case of technical problems the firmware version is an important piece of information for Roche Diagnostics customer support. Sometimes problems can be solved by applying a firmware update. The **About** screen provides such information.

- 1 Select **About** and press the Enter button  to open the corresponding menu.

The serial number, firmware version and the date of the initial setup are displayed.

- 2 Press the Enter button  to switch to the next screen displaying the data encryption key and the BLE (*Bluetooth*[®] Low-Energy) device address.
- 3 Press the Back button  to exit this screen.
- 4 Press the Back button  again to go back to the **Other** menu.

Clear Memory

NOTICE

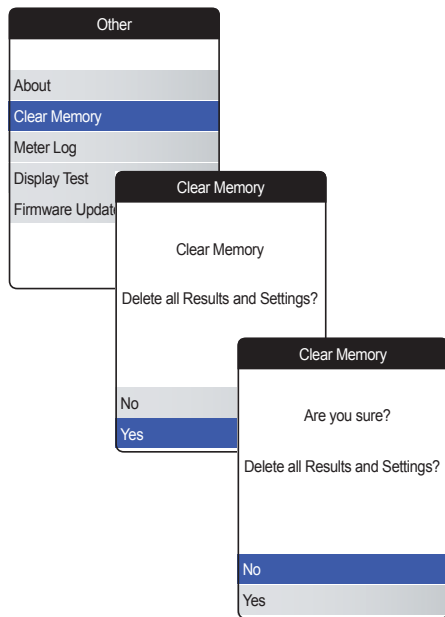
Potential loss of data

The **Clear Memory** function deletes all data (stored test results) and settings from the meter. If you want your data preserved, make sure you have uploaded them to a computer (using the data transfer function) before clearing the memory.

The following data will be cleared:

- Test results
- Stored code chips
- Reminders


The meter will be reset to the default settings. The meter turns off. When the meter is turned on again, you will first need to perform an initial setup (see page 31).

For clearing the memory:

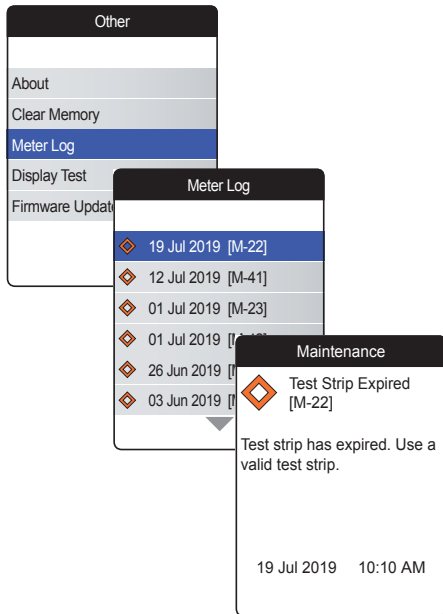- 1 Select **Clear Memory** and press the Enter button to proceed.
- 2 Select **YES** and press the Enter button to proceed.
- 3 Select **YES** again and press the Enter button to clear all data from memory.


Meter Log

The **Meter Log** is another helpful source of information when contacting Roche Diagnostics customer support. The meter stores a list of all warnings, maintenance and error messages.

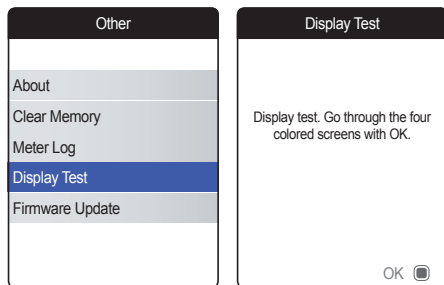- 1 Select **Meter Log** and press the Enter button  to proceed.
- 2 Select the event for which you want view the details and press the Enter button  to proceed.
- 3 When you are finished, close the detail screen with **OK**.

Display Test

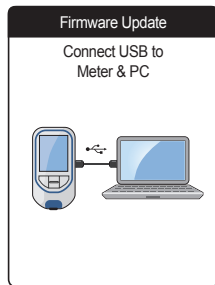
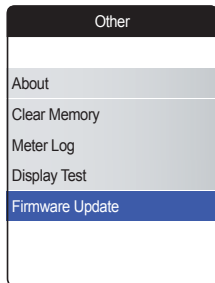
The **Display Test** lets you check the meter's screen for proper functioning. If you suspect the screen (or parts of it) is defective, perform this display test.

- 1 Select **Display Test** and press the Enter button  to proceed.
- 2 Read the information screen and press the Enter button  to proceed.
- 3 Proceed through the following screens by pressing the Enter button , while inspecting each one for defective pixels or other defects.

If all screens appear flawlessly, your display is functioning properly.

If the display is not functioning properly, do not perform further tests and contact your local Roche Diagnostics customer support.


Firmware Update

Performing a firmware update requires an update file (provided by Roche Diagnostics) to be available on your computer.

- 1 Select **Firmware Update** and press the Enter button  to start the update process.
- 2 Connect the meter to the computer via USB cable.

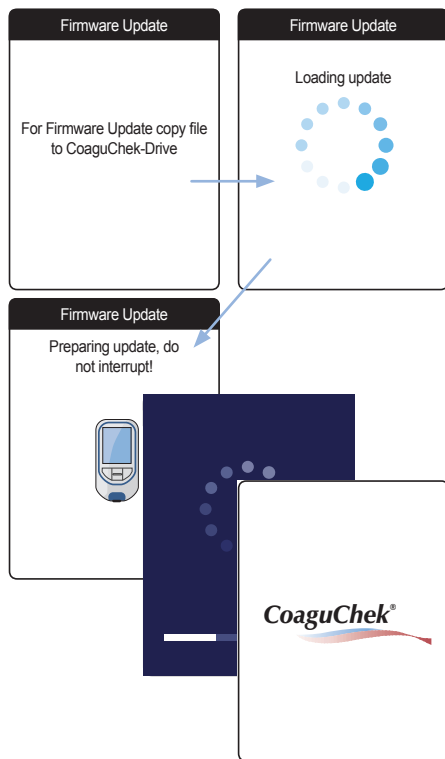The meter will now be mounted on the computer as an external drive (like a USB stick).

- 3 On the computer, open a window showing the content of this drive (CoaguChek).
- 4 Copy (or save) the required update file to this drive.

The meter automatically recognizes the presence of an update file and starts performing the firmware update. Several screens inform you about the progress during installation. When the update has been completed successfully, the CoaguChek logo is displayed. Unplug the USB cable from the meter and the computer.

Troubleshooting

The CoaguChek Vantus meter continually checks its systems for unexpected and unwanted conditions.

Depending on the circumstances, a message may appear on the display of the meter (see chapter *Messages* starting on page 135). These messages are marked with an icon and are accompanied by a description and (if necessary) a possible solution.

Take the action suggested on screen to resolve the problem. If the problem (and the corresponding message) disappears, you may continue using the meter as desired. If the problem persists, contact Roche Diagnostics (see page 154).

For troubleshooting problems which are not indicated by a message see the following pages.

Errors and unusual behavior without error messages

Display/symptom	Possible solution
Meter does not power on (either when pressing the Enter button or when inserting a test strip), display remains dark	Insert new batteries. Always replace all 4 batteries at the same time. Check if the contacts are intact.
Meter displays an unexpected result	Refer to the package insert for the test strips.
Meter does not make sounds	Check the settings for <i>Beeper Volume</i> (page 97) and <i>Button Click</i> (page 99)
Test strip cannot be inserted	Clean and disinfect the meter as indicated on page 103.

If the problem persists, contact Roche Diagnostics (see page 154).

Symbols, Icons, Messages

Symbols on packaging and meter

On the packaging and on the type plate of the meter you may encounter the following symbols and abbreviations shown here with their meaning:

Symbol	Meaning
	Use by
	Batch code/ Lot number
	In vitro diagnostic medical device
	Catalogue number
	Global Trade Item Number
	Serial number
	Consult instructions for use
	Caution, consult accompanying documents. Refer to safety-related notes in the manual accompanying this instrument.
	Manufacturer
	Date of Manufacture
	Temperature limitation (Store at)
	The system fulfills the Canadian and U.S. safety requirements in accordance with CAN/CSA C22.2 No. 61010-1 and UL 61010-1.

Symbol	Meaning
	The compliance mark indicates that the product complies with the applicable standard and establishes a traceable link between the equipment and the manufacturer, importer or their agent responsible for compliance and for placing it on the Australian and New Zealand market.
Rx only	Caution: Federal law restricts this device to sale by or on the order of a physician.

On the meter buttons the following symbols are used:

Symbol	Meaning
	Back button, used to exit the currently displayed menu or go back one step.
	Up/down arrows button, used to select elements, scroll up/down in a screen, or change numerical values.
	Enter button, used to confirm the currently selected option or perform a function. Also used to power the meter on and off.

Icons on the display

Symbol	Meaning
	Indicator for battery capacity (full, almost empty, empty), shown in main menu
	Bluetooth logo, shown when starting a wireless data transfer and in data transfer setup
	USB symbol, shown when starting a wired data transfer and in data transfer setup
	<i>Test</i> icon, shown in main menu and reminders
	<i>My Results</i> icon, shown in main menu
	<i>Reminder</i> icon, shown in main menu
	<i>Medication</i> icon, shown in reminders

On the meter display the following icons may appear.

Symbol	Meaning
	<i>Appointment</i> icon, shown in reminders
	<i>Comment</i> icon, shown with stored test results
	<i>Upload</i> icon, shown with stored test results after data transfer
	Radio button, shows the status of a selectable option (off/on) in meter setup screens
	<i>Settings</i> icon, shown in main menu, and selected option (e.g. comment list)
	<i>Other</i> icon, shown in main menu and reminders

Symbol	Meaning
	<i>Warning message</i>
	<i>Error message</i>

Symbol	Meaning
	<i>Maintenance message</i>
	<i>Information message</i>

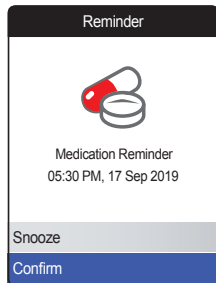
Messages

There are several kinds of messages which can appear on the meter:

- Reminders
- Information
- Maintenance
- Warnings
- Error messages

The following section explains what you can do when such a message appears.

Reminders


Reminders are a useful tool to schedule tests and other events. When a reminder becomes active, it automatically powers on the meter, plays the corresponding sound and displays the reminder message (test, medication, appointment or other). You have two options:

- Select **Confirm**, and press the Enter button to confirm that you will perform the required action **right now**. The reminder will disappear and not be displayed again.
Or:
- Select **Snooze**, and press the Enter button to postpone the reminder message for 5 minutes. The meter powers off automatically. After 5 minutes it powers on and displays the reminder once again.

- When you have snoozed a reminder for the 3rd time, it will not be automatically displayed again. It will be displayed only when you power on the meter the next time.
- When a reminder becomes active while you are using the meter, it will not be displayed or interrupt your current operation (e.g. performing a test or reviewing results). Instead it will be displayed when you power off the meter.

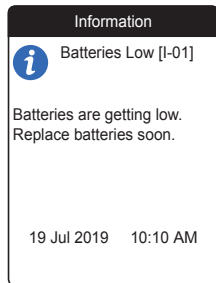Information

An information message informs you about:

- what is happening on the meter
- an existing problem or
- an upcoming problem which requires you to do something soon.

You can continue to use the meter.

- 1** Read the whole information message.
- 2** Press the Enter button  to confirm the message.
- 3** Proceed with the current task.

These information messages may appear on the display. They have the meanings shown:

Information

 **Batteries Low [I-01]**

Batteries are getting low. Replace batteries soon.

19 Jul 2019 10:10 AM

Information

 **Meter Lifetime [I-02]**

Only ~350 tests left before lifetime end. Contact your healthcare professional or Roche Diagnostics.

19 Jul 2019 10:10 AM

Information

 **Wrong Code Chip [I-03]**

Wrong code chip is in the meter. Insert correct code chip.

19 Jul 2019 10:10 AM

Information

 **Reminder in the Past [I-04]**

The date for this reminder is in the past and will not be applied.

19 Jul 2019 10:10 AM

Information

 **Test Strips Expire Soon [I-06]**

Only 99 days left until test strips expire. Make sure you have a new test strip vial available.

19 Jul 2019 10:10 AM

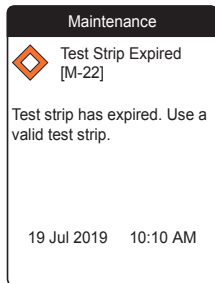
Information

 **Target/Trend only with INR [I-07]**

Target and Trend graphs are only available with unit INR. Change unit to INR to enable this function.

19 Jul 2019 10:10 AM

Maintenance

A maintenance message requires your immediate attention. The meter may stop working temporarily. Some examples:

- The batteries are empty.
- The test strip has expired.
- The meter may detect a problem when performing a self-check.

Almost always you can fix the problem by following the instruction in the maintenance message. You can insert new batteries or start the test again with a new test strip. Or the meter often fixes a problem itself when you power it off and on again.

- 1 Read the whole maintenance message and the suggested action.
- 2 Press the Enter button  to confirm the message.
- 3 Take the action suggested in the message.

These maintenance messages may appear on the display. They have the meanings shown:

<p>Maintenance</p> <p> Invalid Date [M-21]</p> <p>Date is invalid. Enter correct time and date.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> Test Strip Expired [M-22]</p> <p>Test strip has expired. Use a valid test strip.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> Test Strip Error [M-23]</p> <p>Barcode on test strip cannot be read. Repeat test with a new test strip.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> Test Strip Error [M-24]</p> <p>Meter does not recognize test strip. Repeat test with a new test strip.</p> <p>19 Jul 2019 10:10 AM</p>
<p>Maintenance</p> <p> Invalid Code Chip [M-25]</p> <p>Code chip is invalid. Replace with a valid code chip.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> Test Strip Error [M-26]</p> <p>Test strip removed during test. Repeat test with a new test strip.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> Cover Open [M-27]</p> <p>Test strip guide cover removed during test. Close cover and repeat test.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Maintenance</p> <p> USB cable / light interference [M-28]</p> <p>Connected USB cable or external light source interfering with test. Repeat test after disconnecting USB cable and protecting from light source.</p> <p>19 Jul 2019 10:10 AM</p>

Maintenance

Batteries Empty [M-31]

Batteries are nearly empty.
Replace batteries now.

19 Jul 2019 10:10 AM

Maintenance

Ambient Temperature [M-32]

Meter too cold or too warm.
Power meter off. Repeat test
after 30 minutes at room
temperature.

19 Jul 2019 10:10 AM

Maintenance

No Blood Detected [M-41]

No blood was detected.
Repeat test, with a new test
strip if necessary.

19 Jul 2019 10:10 AM

Maintenance

Sample Too Small [M-42]

Not enough sample was
applied. Repeat test with a
new test strip.

19 Jul 2019 10:10 AM

Maintenance

Dosing Error [M-43]

A dosing error was detected.
Repeat test with a new test
strip.

19 Jul 2019 10:10 AM

Maintenance

Test Strip Error [M-44]

Test strip failed internal
control. Repeat test with a
new test strip.

19 Jul 2019 10:10 AM

Maintenance

USB Error [M-71]

A USB error has occurred.
Power meter off and on and
repeat connecting USB.

19 Jul 2019 10:10 AM

Maintenance

Bluetooth Error [M-72]

Pairing has failed. Repeat
pairing process.

19 Jul 2019 10:10 AM

Maintenance

Bluetooth Error [M-73]

A Bluetooth error has occurred. Power meter off and on and repeat data transfer.

19 Jul 2019 10:10 AM

Maintenance

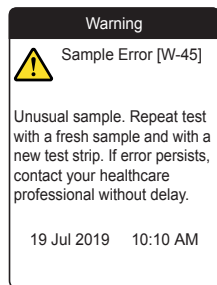
Update Error [M-74]

Software update has failed.
Repeat update process.

19 Jul 2019 10:10 AM

Warnings


A warning message draws your attention to a hazardous situation that could be a risk to your health if not corrected. The CoaguChek Vantus meter will rarely display a warning message. However, if you do receive a warning message, pay close attention to it.

- 1 Read the whole warning message and the suggested action.
- 2 Press the Enter button  to confirm the message.
- 3 Take the action suggested in the message.

NOTE: There is only this warning message at the moment. Therefore, no list required.

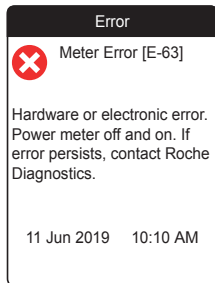Error messages

An error message appears if the meter has a problem that needs your attention and you might not be able to solve. The meter stops working.

- 1 Read the error message.
- 2 Take the action suggested in the message.

If the error persists, please contact your local Roche Diagnostics customer support (see page 154).

These error messages may appear on the display. They have the meanings shown:

<p>Error</p> <p> Meter Lifetime End [E-33]</p> <p>Maximum number of tests has been reached. Contact your healthcare professional or Roche Diagnostics.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-51]</p> <p>Meter self-check detected a general error. Power meter off and on. If error persists, contact Roche Diagnostics.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-52]</p> <p>Temperature or heater error. Please try again in 3:15 minutes. If error persists, contact Roche Diagnostics.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-61]</p> <p>Meter is defective. No test is possible. Contact Roche Diagnostics.</p> <p>19 Jul 2019 10:10 AM</p>
<p>Error</p> <p> Meter Error [E-62]</p> <p>Temperature or heater error. Power meter off and on. If error persists, contact Roche Diagnostics.</p> <p>19 Jul 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-63]</p> <p>Hardware or electronic error. Power meter off and on. If error persists, contact Roche Diagnostics.</p> <p>11 Jun 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-64]</p> <p>Test strip reader error. Power meter off and on. If error persists, contact Roche Diagnostics.</p> <p>11 Jun 2019 10:10 AM</p>	<p>Error</p> <p> Meter Error [E-65]</p> <p>Code chip cannot be read. Replace with a different code chip.</p> <p>11 Jun 2019 10:10 AM</p>

Error	Error	Error
 Meter Error [E-66]	 Meter Error [E-67]	 Power Failure [E-68]
A software error has occurred. Power meter off and on. If error persists, contact Roche Diagnostics.	A memory error has occurred. Power meter off and on. If error persists, contact Roche Diagnostics.	There was an unexpected interruption of the meter's power supply. Repeat your last procedure.
11 Jun 2019 10:10 AM	11 Jun 2019 10:10 AM	11 Jun 2019 10:10 AM

This page intentionally left blank.

General Product Information

Ordering

Please contact your pharmacist, testing service provider or specialist supplier.

Item	REF/Catalog Number
CoaguChek XS PT Test, 6 test strips	REF 0 4625374 160
CoaguChek Softclix Lancet, 25 lancets	REF 0 6979009 001

Product limitations

Please read the information in the package insert supplied with the test strips for detailed product data and limitations.

WARNING

If you receive a discrepant INR result from another device, please contact your healthcare professional.

Operating conditions and technical data

Temperature range	+59 °F to +90 °F
Relative humidity	10 to 85% (no condensation)
Maximum altitude	13120 feet
Measuring range	INR: 0.8–6.0
Memory	400 test results with date and time
Interfaces	USB type B <i>Bluetooth</i>
Battery operation	4 x 1.5 V batteries (alkali-manganese) type AAA (LR03)
Safety class	III
Automatic power-off	After 2 minutes
Dimensions	approx. 145 x 75 x 30 mm
Weight	135 g (without batteries)

Sample material

For information regarding sample type, sample size and interferences refer to the respective test strip package insert.

Storage and transport conditions

Temperature range

Short-term storage (meter only) -13 °F to +158 °F

Long-term storage (meter only) +23 °F to +113 °F

Test strips Refer to the test strip package insert

Relative humidity 10 to 85% (no condensation)

Disposal of the CoaguChek Vantus meter

During measurements the meter may come into contact with blood. Used meters therefore carry a risk of infection. Please dispose of your used meter – after removing the batteries and cleaning the meter – according to the regulations applicable in your country. For information about correct disposal please contact your local council.

For healthcare professionals

Dispose of the meter in line with your facility's procedures for dealing with potentially biohazardous waste.

Disposal of used batteries

Do not dispose of the batteries with normal domestic waste.

Dispose of used batteries in an environmentally responsible manner and in accordance with applicable local regulations and directives. Contact your local council / local authority or the manufacturer of the used batteries for advice on correct disposal.

Repairs

Please note that repairs and other modifications to the meter may only be performed by persons authorized by Roche Diagnostics.

Information service

If you have any further questions, please contact your local customer support and service center.

USA

Roche Diagnostics
Indianapolis, IN 46256

Please call CoaguChek Technical Services at 1-800-428-4674 if you have questions regarding the handling of your meter, the reliability of your results, or if you suspect the meter is defective.

Product Safety Information

***Bluetooth* Wireless Technology**

The meter can utilize *Bluetooth* wireless technology to communicate and transfer information to other devices (e.g., computer or smartphone).

Bluetooth wireless technology is a form of radio frequency (RF) technology that operates in the unlicensed industrial, scientific, and medical band at 2.5 GHz. The meter can only communicate with the devices it is paired with; therefore, other *Bluetooth* wireless technology devices cannot access your personal information on the meter.

Radio Frequency Communication

In the event that there is interference from another device, it is recommended that you increase the distance between the meter and that device. You could also turn off the interfering device. In addition, you could turn off Bluetooth wireless technology on the meter.

Changes or modifications to the device not expressly approved by Roche Diagnostics could void the user's authority to operate the device.

The device has been tested and found to comply with the limits for a Class B digital device. The device generates, uses, and can radiate radio frequency energy.

Electromagnetic Compatibility

This meter meets the electromagnetic immunity requirements as per IEC 61326-2-6. The chosen basis for electrostatic discharge immunity testing was basic standard IEC 61000-4-2. In addition, it meets the electromagnetic emissions requirements as per IEC 61326-2-6. Its electromagnetic emission is thus low. Interference from other electrically driven equipment is not to be expected.

Limited One (1) Year Warranty

Roche Diagnostics warrants, to the original purchaser only, that the meter shall be free from defects in material and workmanship for a period of one (1) year from the date of purchase.

Purchaser's sole and exclusive remedy, with respect to the CoaguChek Vantus meter and parts thereof, shall be the repair and/or replacement of the meter or parts at the option of Roche Diagnostics.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE DIAGNOSTICS MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR USE OF THE METER. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY, IS IMPLIED FROM THE SALE OF THE COAGUCHEK VANTUS METER. NO WARRANTY, EXPRESS OR IMPLIED (IF ANY), SHALL EXTEND FOR A LONGER DURATION THAN THE DURATION OF THE EXPRESS WARRANTY STATED ABOVE.

The foregoing warranty shall not apply to a meter which is damaged by accident or subject to alteration, misuse, tampering, and/or abuse, including cleaning or disinfecting the meter with any product other than products approved for such purpose in the User's Manual. Meters which show damage or misuse will be handled in accordance with the non-warranty service policy of Roche Diagnostics.

The warranty of any repaired/replacement meter will expire on the date of the original warranty expiration or ninety (90) days after shipment of a replacement system, whichever period is longer.

Index

- A**
Altitude..... 18
- B**
Battery
 compartment cover..... 23
 insertion 28–29
 low..... 45
 power level..... 24
 type 23, 28
Bluetooth
 pairing..... 101
Button
 back..... 22
 click..... 99
 enter..... 22
 up/down..... 22
- C**
Cleaning..... 103–120
Code chip
 insertion 36–38
 slot..... 23
- D**
Data transfer 66–70
 Bluetooth 69–70
 setup..... 100
 USB 67–68
Display..... 22
 brightness..... 98
 test..... 126
- E**
Electromagnetic interference 18
- F**
Firmware..... 122, 127
- H**
Humidity..... 18
- I**
Icons..... 133–134
Information Service..... 154

M

Measuring range.....	56
Memory	
clearing.....	124
Messages	
error.....	145
information.....	138
reminder.....	136
warnings.....	144
Meter	
cleaning.....	103–120
overview.....	22–23
powering off.....	35
powering off (automatically) ...	24
powering on.....	31
settings.....	87–102
Meter log.....	125

O

Operating conditions.....	18
Other (functions).....	121–126

P

Power

off.....	35
on.....	31

Q

Quality control.....	19
----------------------	----

R

Reminder.....	136
edit/delete.....	86
new.....	82
setting up.....	81–86
Reports.....	60–64
embedded.....	71–80
exporting to *.csv.....	79
list.....	77
printing.....	78
targets.....	64–65, 76
trends.....	61–62, 75
Result.....	51
comment.....	52–53, 59
display.....	51
format.....	93
review.....	58–59

S	
Setup.....	87–102
beeper volume.....	97
button click.....	99
data transfer.....	100–101
display brightness.....	98
initial.....	31–33
language.....	90
result format.....	93
set targets.....	94–95
settings overview.....	88–89
time and date.....	33, 91
Symbols	
battery.....	24
on packaging and meter	131–132
on the display.....	133–134
T	
Temperature.....	18, 40
Test.....	39–56
blood sample.....	42, 48–50
performing.....	44
preparing.....	43
result display.....	51
Test strip	
insertion.....	46
Test strip guide.....	22
U	
USB	
port.....	23

This page intentionally left blank.

This page intentionally left blank.

This page intentionally left blank.

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©2018-2019 Roche Diagnostics GmbH
Manufactured for and distributed in the U.S.A. by: